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English

IMPORTANT! Read the information contained in these operating instructions carefully before using the product.

In accordance with the EN ISO 17664 Normative, Nueva Galimplant will provide all of the information necessary to process their medical devices labelled as "Non sterile" and which need to be cleaned, disinfected and sterilized prior to use, both for the first use and for all subsequent usages.

Warnings:

- Do not use the products without having carried out the previous cleaning, disinfection and sterilization.
- For their own safety, staff should use adequate protective clothing and equipment at all times: gloves, mask, protective glasses, etc;
- Nueva Galimplant recommends the use of aldehyde free cleaner and disinfectants with proven effectiveness, in order to avoid the fixation of blood proteins (included on the VAH/DGHM list or which have the CE Marking).
- The use of neutral or slightly alkaline cleaning agents is recommended by Nueva Galimplant.
- It is essential to verify that the cleaning product used is compatible with the medical products are going to be processed and respect the specifications of use (temperature, concentration and time) indicated by the manufacturer.
- Nueva Galimplant cannot control the clinical manipulation procedures, the cleaning methods or any other user level conditions, therefore they will not be held liable for the sterilization of the products even when the recommended guidelines contained in this instruction manual have been followed.

Limitations and restrictions:

The repetition of the indicated processes has a minimal effect on the galimplant® products. The useful life of these is normally determined by the wear and tear and damage caused due to repeated use.

The best way to assess the state of deterioration is by carrying out a thorough and functional check prior to use.

Preparation in the place of use

As soon as possible following their use, the galimplant® products should be transferred to the place where they are to be cleaned, in order avoid the drying-on of any blood residues, tissues, secretions or bone fragments. If it is likely that the cleaning process will be delayed, these should be covered with a moist cloth or stored in an airtight container filled with water in order to delay the drying-on of the organic material and to avoid the risk of contamination.

Preparation before cleaning

- As soon as possible following their use, immerse the galimplant® products in a container that contains a cleansing solution with proteolytic enzymes, prepared in accordance with the manufacturer's specifications, respecting the temperature, concentration and immersion time. These enzyme solutions separate proteins and ensure that blood and other secretions do not dry on the products.
- Make sure that all the wrench's surfaces are covered by the solution and that the instruments do not touch each other.
- Manually brush with a nylon bristle brush (never metal, wool or steel brushes) until all of the visible dirt and residue has been eliminated, paying special attention to the zones which may not be reached by the brushing action, such as grooves or internal parts.
- Remove the products from the container containing solution and rinse them thoroughly with plenty of water, preferably purified water, to ensure the reduction of organic material from mechanical drag.

Cleaning and disinfection

- Place the galimplant® products in the thermal washer-disinfector's tray or basket. Nueva Galimplant S.L.U. recommends thermal washer-disinfectors that comply with the EN ISO 15883 normative.
- Run the cleaning, disinfecting and drying cycle. Nueva Galimplant has validated the Vario-TD program by the professional brand Miele, using the detergent neodisher® MediClean forte diluted at 0.5%. This program consists of:
 - Cold water soak for 4 minutes.
 - 55° C clean for 10 minutes with detergent.
 - Cold water neutralisation for 6 minutes.
 - Cold water mid-cycle rinse for 3 minutes.
 - 93° C thermal disinfection for 5 minutes.
 - 110° C dry for 25 minutes.

To guarantee optimum protection of the attachments we recommend that purified water is used throughout the whole process.

Note: it is important to always follow the instructions provided by the detergent and thermal washer-disinfector manufacturers.

Packaging

Once they are clean, disinfected and completely dry, the products must be packaged in a sterilisation bag.

The packaging of thermally sterilised medical products must comply with the following requisites:

- Be appropriate for steam sterilisation, that is, have a minimum thermic resistance of 137° C and be sufficiently permeable for steam.
- Protect the individual products or surgical boxes sufficiently from mechanical damage.
- Comply with the EN ISO 11607 normative.

Once the products have been packaged a label should be added to each sterilisation bag which contains data such as the sterilisation date and it should include the reference, batch and the number of products if the bag does not have a transparent window which allows the user to see what is inside.

Sterilisation

- Load the autoclave with the conditioned galimplant® products in the sterilisation bags. Do not overload the autoclave and always follow the equipment manufacturer's instructions.
- Just like in the other steps of these instructions, we recommend the use of purified water to guarantee optimum quality.
- Nueva Galimplant recommends using an autoclave which complies with the requisites of the EN ISO 17665 normative, in addition to being validated and having adequate maintenance. The validation of the sterilisation was carried out in a steam autoclave using a 134° C pre-vacuum cycle for 6 minutes.
- To control the sterilisation process, the use of chemical indicators is recommended, such as, for example, the verification strips which the sterilisation bags come equipped with and which change colour when the sterilisation process has finished.

Note: it is important to always follow the autoclave manufacturer's instructions and keep maintenance up to date.

Additional information

For more information concerning the use of galimplant® products, contact your local representative or visit our website www.galimplant.com.

Validity

These instructions for use will be valid as of May 2021.

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