**Español**

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**Safety, responsibility and guarantee**

We guarantee the quality of our products within the framework of our sales and supply conditions. This product is part of a global concept and should only be used in conjunction with the associated original products according to the Nueva Galimplant instructions and recommendations.

The non-recommended use of products manufactured by third parties together with galimplant® products will void any guarantee or other obligation, express or implicit, by Nueva Galimplant, S.L.U.

The user has sole responsibility for the procedure and use of the product. It is the practitioner's responsibility to use the product in accordance with the current instructions for use and to determine if it is appropriate for each patient's individual situation. In particular, this responsibility is expressly applicable in the case of its use in non-recommended procedures. If in doubt, the user should consult us.

**Assistance and training**

Technical assistance concerning the use of our products will be carried out verbally, by written means, by electronic means or by demonstrations. Said assistance will be given according to our knowledge of

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#### Manual Method

- Place the galimplant® surgical drills in a perforated tray or basket.
- Prepare an ultrasonic bath and a detergent and disinfectant solution in accordance with the manufacturer's specifications of the product used.
- Nueva Galimplant has validated this process using the product neodisher® septo PreClean diluted at 1% to prepare the ultrasonic bath.
- Immerse the basket with the galimplant® surgical drills in the ultrasonic bath and keep them at a temperature of 20° C for 15 minutes.
- Remove the drills from the ultrasonic bath and rinse with plenty of purified water.
- If necessary, repeat the cleaning and disinfection step.
- Dry the devices using filtered compressed air and single use lint free cloth.

#### Automatic Method

- Place the galimplant® surgical drills in the thermal washer-disinfectors tray or basket. Nueva Galimplant S.L.U. recommends thermal washer-disinfectors that comply with the EN ISO 15883 normative.
- Run the cleaning, disinfecting and drying cycle.
  - Nueva Galimplant has validated the Vario-TD program by the professional brand Miele, using the detergent neodisher® MediClean forte diluted at 0.5%. This program consists of:
    - Cold water soak for 4 minutes.
    - 55° C clean for 10 minutes with detergent.
    - Cold water neutralisation for 6 minutes.
    - Cold water mid-cycle rinse for 3 minutes.
    - 93° C thermal disinfection for 5 minutes.
    - 110° C dry for 25 minutes.

To guarantee optimum protection of the attachments we recommend that purified water is used throughout the whole process.

Note: it is important to always follow the instructions provided by the detergent and thermal washer-disinfector manufacturers.

#### Inspection, maintenance and testing

- Carefully inspect all of the galimplant® surgical drills in order to check if any damage has been caused to them (corrosion, loss of shape, breaking) or contamination. We recommend that this inspection is carried out in a well-lit environment or using a magnifying lens in order to improve visibility.
- If there are any traces of contamination on any of the devices the cleaning and disinfection process should be repeated.
- If there are any damage (deformation, failure of the drill, loss cutting capacity...), the drill should be discarded.

#### Packaging

Once they are clean, disinfected and completely dry, the drills must be packaged in a surgical box and then the box should be packaged in a sterilisation bag.

The packaging of thermally sterilised medical devices must comply with the following requisites:

- Be appropriate for steam sterilisation, that is, have a minimum thermic resistance of 137° C and be sufficiently permeable for steam.

- Protect the individual devices or surgical boxes sufficiently from mechanical damage.

- Comply with the EN ISO 11607 normative.

Once the devices have been packaged a label should be added to each sterilisation bag which contains data such as the sterilisation date and it should include the reference, batch and the number of devices if the bag does not have a transparent window which allows the user to see what is inside.

#### Sterilisation

- Load the autoclave with the conditioned galimplant® surgical drills in the sterilisation bags. Do not overload the autoclave and always follow the equipment manufacturer's instructions.

Just like in the other steps of these instructions, we recommend the use of purified water to guarantee optimum quality.

Nueva Galimplant recommends using an autoclave which complies with the requisites of the EN ISO 17665 normative, in addition to being validated and having adequate maintenance.

The validation of the sterilisation was carried out in a steam autoclave using a 134° C pre-vacuum cycle for 6 minutes.

- To control the sterilisation process, the use of chemical indicators is recommended, such as, for example, the verification strips which the sterilisation bags come equipped with and which change colour when the sterilisation process has finished.

Note: it is important to always follow the autoclave manufacturer's instructions and keep maintenance up to date.

#### WARNINGS

Use the surgical drills only for the insertion of original galimplant® dental implants.

The practitioner must provide the patient with the information about side effects, interactions and complications in relation to galimplant® surgical drills.

The clinical practitioner must verify that the materials and techniques to be used during surgical procedure are appropriate and compatible with the galimplant® surgical drills and must follow the manufacturer's instructions for use of these materials.

Failure to observe the procedures set out in these instructions could harm the patient and give rise to one or all of the following complications:

- Aspiration or swallowing of the components.
- Infection.
- Damage to the implant or instruments.
- Implant failure.

In the event of a serious incident related to the use or result of use of the product, inform Nueva Galimplant ([www.galimplant.com](http://www.galimplant.com)) and your National Competent Authority.

#### Procedures for the disposal of the product

If used, dispose the surgical drills as biological waste, according to the local regulations.

#### Validity

These instructions for use will be valid as of May 2022.

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