**IMPLANTE DENTAL ESPECIAL ONE-PIECE
ONE-PIECE SPECIAL DENTAL IMPLANT
IMPLANTE DENTARIO ESPECIAL ONE-PIECE**
Español

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English

IMPORTANT! Read the information included in these instructions for use before using the product

NUEVA GALIMPLANT complies with all the regulations established by the European laws and directives relating to the manufacture and distribution of medical products. The Galimplant® implant system, attachments and instruments are certified and authorised for sale by a Notified Body, guaranteeing the perfect quality of all its products. The use and application of the Galimplant® System is not under the manufacturer's control, and the user is responsible for any damages which may occur due to an incorrect manipulation or use of the product.

Safety, responsibility and guarantee

We guarantee the quality of our products within the framework of our sales and supply conditions.

This product is part of a global concept and should only be used in conjunction with the associated original products according to the Nueva Galimplant instructions and recommendations. The non-recommended use of products manufactured by third parties together with galimplant® products will void any guarantee or other obligation, express or implicit, by Nueva Galimplant, S.L.U.

The user has sole responsibility for the procedure and use of the product. It is the practitioner's responsibility to use the product in accordance with the current instructions for use and to determine if it is appropriate for each patient's individual situation. In

particular, this responsibility is expressly applicable in the case of its use in non-recommended procedures. If in doubt, the user should consult us.

Assistance and training

Technical assistance concerning the use of our products will be carried out verbally, by written means, by electronic means or by demonstrations.

Said assistance will be given according to our knowledge of the current state of science and techniques when the product is on sale. This does not exempt the user from their obligation to personally make sure that the product is appropriate for the purposes and procedures planned.

Nueva Galimplant, with the aim of ensuring the correct use of its products, offers courses and practical sessions about its products through the platform: galimplant training.

Precautions and warnings

The galimplant® dental implants are delivered sterile; they are for single use and must not be used after their expiration date.

Warning! Re-sterilisation and/or reuse of implants may result in the risk of crossed contamination.

The following descriptions are insufficient for the immediate application of any Galimplant® system implants. This system should only be used by dentists, doctors and surgeons. The dentist/doctor/surgeon must have knowledge in oral implantology and experience in the manipulation of the product in order to use them safely and correctly following the indications of these instructions for use.

Description

The one-piece implants are manufactured using commercially pure grade IV titanium,

- Use the appropriate drilling speed for each drill diameter. A speed of between 300 and 800 rpm is recommended.
 - Use sharpened surgical instruments.
 - An intermittent drilling technique should be applied.
 - The drills should be highly refrigerated using pre-cooled (5°) sterile physiological saline solution (NaCl).
 - Use drills with increasing diameter: first the lance drill and then the Ø 2mm drill. To favour the primary stability of the implant, the diameter of the last drill should be less than the diameter of the implant.

• Extraction of the implant from the packaging and insertion into the bed.

The placement of the implant once the implant bed has been prepared may be done manually, using the manual wrench or mechanically using the contra-angle or the torque wrench. When mechanically placing one-piece special implants, the speed should not exceed 30 rpm and the insertion torque should be between 25-30Ncm.
 To take the implant out of the container open the external packaging, unscrew the plastic top and remove the operculum. The implant may be taken out of the package in the following ways:
 1. Connect the manual driver to the metal implant transfer and extract the plastic holder from the container.
 2. Insert the motor wrench in the contra-angle, connect the point to the metal implant transfer and extract the plastic holder from the container.
 Once connected remove the implant from the plastic holder by applying light pressure in an upwards direction at the aperture where it is placed and hold it with any of the assembly techniques indicated above to bring it to the mouth and insert it into the implant bed. Once the implant has been inserted, the implant transfer must be removed by pulling gently.

Documentation, traceability and implant card.

Nueva Galimplant S.L.U. recommends keeping an exhaustive clinical, radiological and photographic documentation. All products can be identified through their reference and batch code and / or UDI code. To maintain the mandatory traceability of these products, two adhesive labels are included, one for the medical record and the other for the patient, the latter must be attached to the implant card supplied with the products and which must be delivered to the patient fully completed once the surgery is over.

WARNINGS

The physician should provide the patient with information regarding the contraindications, side effects, interactions and complications associated with the galimplant® implants.

The patient should avoid activities which require a high level of physical effort immediately after the insertion of the dental implant.

When using our products, the necessary measures should be taken to avoid inhalation. Before placing the implant ensure that there is no damage has been caused to both containers. A damaged container may mean that the implant is now not sterile. When removing the implant from the sterile container follow the corresponding aseptic principles. The container should only be opened when the surgical intervention is to be carried out.

The health professional must ensure that the materials and techniques that are to be used during the clinical and prosthetic procedures are appropriate and compatible with the one-piece special implants and the instructions for use provided by the manufacturer of these materials should be followed.

For the preparation of the implant bed, and the placement and restoration of the one-piece special implants only original galimplant® products must be used.

Failure to follow the procedures summarised in these instructions may cause damage to the patient and may lead to one or more of the following complications.

- Inhalation or swallowing of the components.
- Infection.
- Damage to the implant, abutment, components or instruments.

In the event of a serious incident related to the use or result of use of the product, inform Nueva Galimplant (www.galimplant.com) and your National Competent Authority.

Procedures for the disposal of the product

Once extracted from the oral cavity the prosthetic components must be disposed of as biological waste in accordance with the local regulations currently in force.

Summary of Safety and Clinical Performance

The Summary of Safety and Clinical Performance (SSCP) for the product is available in the European database EUDAMED. The document can be obtained from the website: <https://ec.europa.eu/tools/eudamed>.

Validity

These instructions for use will be valid as of May 2022.

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