Español

Pilares Protésicos Galimplant

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IMPORTANT! Read the information contained in these operating instructions carefully before using the product.

NUEVA GALIMPLANT complies with all the regulations established by the European laws and directives relating to the manufacture and distribution of medical products. The galimplant® implant system, attachments and instruments are certified and authorised for sale by a Notified Body, guaranteeing the perfect quality of all its products. The use and application of the galimplant® system is not under the manufacturer's control, and the user is responsible for any damages which may occur due to an incorrect manipulation or use of the product.

Safety, responsibility and guarantee

We guarantee the quality of our products within the framework of our sales and supply conditions. This product is part of a global concept and should only be used in conjunction with the associated original products according to the Nueva Galimplant instructions and recommendations. The non-recommended use of products manufactured by third parties together with galimplant® products will void any guarantee or other obligation, express or implicit, by Nueva Galimplant, S.L.U.

The user has sole responsibility for the procedure and use of the product. It is the practitioner's responsibility to use the product in accordance with the current instructions for use and to determine if it is appropriate for each patient's individual situation. In particular, this responsibility is expressly applicable in the case of its use in non-recommended procedures. If in doubt, the user should consult us.

Assistance and training

Technical assistance concerning the use of our products will be carried out verbally, by written means, by electronic means or by demonstrations. Said assistance will be given according to our knowledge of the current state of science and techniques when the product is on sale. This does not exempt the user from their obligation to personally make sure that the product is appropriate for the purposes and procedures planned. Nueva Galimplant, with the aim of ensuring the correct use of its products, offers courses and practical sessions about it products through the platform: galimplant.training.

Precautions and warnings

In general terms, our products must be held to avoid their aspiration during intraoral use or insertion in the implant. Galimplant® prosthetic abutments have not been sterilised before delivery, so they must be cleaned, disinfected and sterilised before use and it is essential that they are handled in an aseptic manner. Galimplant® prosthetic abutments are of single use. The reusing of these types of products could lead to a deterioration of their characteristics which involves risk of tissue infection, surgical or prosthodontic failure and/or damage to the patient's health.

The following descriptions are insufficient for the immediate application of any abutment from the galimplant® system, this system should only be used by dentists, doctors and surgeons. The dentist/doctor/surgeon must have knowledge in oral implantology and experience in the manipulation of galimplant® products, in order to use them safely and correctly, following the indications of these instructions for use.

Patient Groups and Clinical Benefits

The galimplant® prosthetic abutments with direct implant connection are intended for use in totally or partially edentulous people or with poor prognosis or non-restorable teeth that request treatment and that meet the criteria for psychological, medical, anatomical, surgical, prosthodontic and maintenance selection specified in these instructions for use. The clinical benefit of treatment is the replacement of the patient's missing teeth.

Straight Abutments

Intended use

Our straight abutments with direct implant connection are solely and exclusively intended to help support cement-restorations with the aim of restoring the patient's mastication function and phonetics.

Indications

- Our straight abutments are recommended in the following cases:
- Single or multi-unit restorations cemented on top of the abutment.

Description

Galimplant® straight abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (internal and external), platforms, diameters and lengths. Our straight abutments are available in the following configurations:

- Non-rotational straight abutment with shoulder heights of 0, 1, 3 and 5mm and golden colour anodized.
- Rotational straight abutment with shoulder heights of 0 mm and golden colour anodized. The shoulder height will depend on the soft tissue height. Supplied both individually and as a kit with 2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic). Recommended torque 30 Ncm. Do not re-use. Single use.

Precautions during use

- Tissue reaction caused by provisional cement.
- Difficulties removing remaining cement.
- Less control of the crown or bridge's settling during the cementing process.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS		External connection	Internal connection
		<i>P: unit sale // EP: kit sale</i>	
Rotational	0mm	PR 4000 EPR 4000	PR 04000 EPR 04000
Non-rotational	0mm	PA 4000 / EPA 4000	PA 04000 / EPA 04000
	1mm	PA 4010 / EPA 4010	PA 04010 / EPA 04010
	3mm	PA 4030 / EPA 4030	PA 04030 / EPA 04030
	5mm	PA 4050 / EPA 4050	PA 04050 / EPA 04050
Compatible Implant Model		ICE IPXE	ICI IPX

Angled Abutments

Intended use

Our angled abutments with direct implant connection are intended to resolve implant angulation problems, helping to support cement-restorations with the aim of restoring the patient's mastication function and phonetics.

Indications

- The angled abutments are recommended in the following cases:
- Single- or multiple-unit cement-restorations.
 - Fixed restorations with non-parallelism between implants.

Description

Galimplant® angled abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (internal and external), platforms, diameters and lengths. Galimplant® angled abutments are available in the following configurations:

- 15° with shoulder heights of 1, 3 and 5 mm and golden colour anodized.
- 30° with shoulder heights of 1, 3 and 5 mm and golden colour anodized.

The amount of degrees will depend on the implant's inclination and the shoulder height will depend on the soft tissue height. Supplied both individually and as a kit with 2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic). Recommended torque 20-30 Ncm. Do not re-use. Single use.

Precautions during use

- Tissue reaction caused by provisional cement.
- Difficulties removing remaining cement.
- Less control of the crown or bridge's settling during the cementing process.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS		External Connection	Internal Connection
Angulation	Height	<i>P: unit sale // EP: kit sale</i>	
15°	1mm	PA 154010 / EPA 154010	PA 0154010 / EPA 0154010
	3mm	PA 154030 / EPA 154030	PA 0154030 / EPA 0154030
	5mm	PA 154050 / EPA 154050	PA 0154050 / EPA 0154050
30°	1mm	PA 304010 / EPA 304010	PA 0304010 / EPA 0304010
	3mm	PA 304030 / EPA 304030	PA 0304030 / EPA 0304030
	5mm	PA 304050 / EPA 304050	PA 0304050 / EPA 0304050
Compatible Implant Model		ICE IPXE	ICI IPX

Angulable Aesthetic Abutments

Intended use

Our angulable aesthetic abutments with direct implant connection are intended to resolve implant angulation problems, helping to support screw-retained restorations with the aim of restoring the patient's mastication function and phonetics.

Indications

- The angulable aesthetic abutments are recommended in the following cases:
- Single- or multiple-unit screw-retained restorations.

Description

Galimplant® angulable aesthetic abutments are made from grade V titanium and can be used in galimplant® internal connection implants with different platforms, diameters and lengths. Galimplant® angulable aesthetic abutments are available in the following configurations:

- Shoulder heights of 2, 3 and 4 mm and golden colour anodized.

The amount of the shoulder height will depend on the soft tissue height. Supplied both individually and as a kit with 2 screws golden colour anodized (one clinical, packed inside a capsule and another prosthetic). Recommended torque 20-30 Ncm. Do not re-use. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS	Internal Connection
Height	<i>P: unit sale // EP: kit sale</i>
2mm	PEA DP 04020 / EPEA DP 04020
3mm	PEA DP 04030 / EPEA DP 04030
4mm	PEA DP 04040 / EPEA DP 04040
Compatible Implant Model	ICI IPX

Straight Multi-position Abutments

Intended use

Our straight multi-position abutments with direct implant connection are designed for single- or multi-unit screw-retained restorations, with the aim of restoring the patient's mastication function and phonetics.

Indications

Our multi-position straight abutments are recommended in fixed single- or multi-unit screw-retained restorations.

Description

Galimplant® multi-position straight abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (external and internal), platforms, diameters and lengths. Our straight abutments are available in the following configurations:

- Internal Connection
 - Non-rotational multi-position straight abutment with shoulder heights of 0, 1, 2 and 3 mm and golden colour anodized.
 - Rotational multi-position straight abutment with shoulder heights of 0, 1, 2 and 3 mm and pink colour anodized.
- External Connection
 - Non-rotational multi-position straight abutment with shoulder heights of 0, 1, 2 and 3 mm and golden colour anodized.
 - Rotational multi-position straight abutment with shoulder heights of 0, 1, 2 and 3 mm and pink colour anodized.

The shoulder height will depend on the soft tissue height. Supplied both individually and as a kit with:

- 2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic)
- 1 transepithelial transfer attachment
- 1 screw for the transepithelial transfer attachment
- 1 transepithelial closing abutment

Recommended torque:

- 30 Ncm multi-position straight abutment.
- 10 Ncm prosthetic screw.

Do not reuse. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURACIONES					
Conexión Externa			Conexión Interna		
MUST: venta unitaria // KMUST: venta en kit					
Altura	Rotacional	Anti-rotacional	Altura	Rotacional	Anti-rotacional
1mm	MUST 4010 KMUST 4010	MUSTA 4010 KMUSTA 4010	1mm	MUST 04010 KMUST 04010	MUSTA 04010 KMUSTA 04010
2mm	MUST 4020 KMUST 4020	MUSTA 4020 KMUSTA 4020	2mm	MUST 04020 KMUST 04020	MUSTA 04020 KMUSTA 04020
3mm	MUST 4030 KMUSTA 4030	MUSTA 4030 KMUSTA 4030	3mm	MUST 04030 KMUSTA 04030	MUSTA 04030 KMUSTA 04030
Modelo Implante Compatible					
ICE IPXE			ICI IPX		

Multi-position Angled Abutments

Intended use

Our multi-position angled abutments with direct implant connection are designed for multi-unit screw-retained restorations, with the aim of restoring the patient's mastication function and phonetics.

Indications

- Our multi-position angled abutments are recommended in the following cases:
- Fixed multi-unit screw-retained restorations.
 - Restorations of the posterior sector.
 - Correction of implants with large non-parallelisms.

Description

Galimplant® multi-position angled abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (external and internal), platforms, diameters and lengths. Our multi-position angled abutments are available in the following configurations:

- Rotational Internal Connection:
 - Multi-position abutment angled at 15° with shoulder heights of 2 and 3 mm and pink colour anodized.
 - Multi-position abutment angled at 30° with shoulder height of 3 mm and pink colour anodized.
 - Multi-position abutment angled at 45° with shoulder height of 5 mm and pink colour anodized.
 - Multi-position abutment angled at 50° with shoulder height of 5 mm and pink colour anodized.
 - Multi-position abutment angled at 60° with shoulder height of 5 mm and pink colour anodized.
- Rotational External Connection:
 - Multi-position abutment angled at 15° with shoulder heights of 2 and 3 mm and pink colour anodized.
 - Multi-position abutment angled at 30° with shoulder height 3 and 4 mm and pink colour anodized.

The shoulder height will depend on the soft tissue height and the angulation will depend on the degree of the implant's inclination. Supplied both individually and as a kit with:

- 1 transporter
- 2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic)
- 1 transepithelial transfer attachment
- 1 screw for the transepithelial transfer attachment
- 1 transepithelial closing abutment

Recommended torque:

- 30 Ncm screw to fix the abutment to the implant
- 10 Ncm prosthetic screw to fix the abutment to the prosthesis

Do not reuse. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS		External Connection	Internal Connection
		<i>MU: unit sale // KMU: kit sale</i>	
Height	Angulation	Rotational	Rotational
2mm	15°	MU 154020 / KMU 154020	MU 1504020 / KMU 1504020
3mm	15°	MU 154030 / KMU 154030	MU 1504030 / KMU 1504030
	30°	MU 304030 / KMU 304030	MU 3004030 / KMU 3004030
4mm	30°	MU 304040 / KMU 304040	MU 3004040 / KMU 3004040
5mm	45°	-	MU 4504050 / KMU 4504050
	50°	-	MU 5004050 / KMU 5004050

	60°	-	MU 6004050 / KMU 6004050
Compatible Implant Model		ICE IPXE	ICI IPX

Aesthetic Straight Multi-position Abutments

Intended use

Our aesthetic straight multi-position abutments with direct implant connection are designed for screw-retained restorations, with the aim of restoring the patient's mastication function and phonetics.

Indications

Our aesthetic straight multi-position abutments are recommended in fixed single- or multi-unit screw-retained restorations with high aesthetic requirements. Our non-rotational multi-position slim straight aesthetic abutments are not recommended for large fixed multi-unit screw-retained restorations.

Description

Galimplant® aesthetic straight multi-position abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (external and internal), platforms, diameters and lengths. Our straight aesthetic abutments are available in the following configurations:

- Internal Connection
 - Rotational aesthetic straight multi-position abutment with shoulder heights of 1, 2, 3, 4 and 5 mm for narrow platform and heights of 2, 3, 4 and 5mm for wide platform and pink colour anodized.
 - Non-rotational aesthetic straight multi-position abutment with shoulder heights of 1, 2, 3, 4 and 5 mm for narrow platform and golden colour anodized.
 - Solidary non-rotational aesthetic straight multi-position abutment with shoulder heights of 1, 2, 3, 4 and 5 mm for narrow platform and heights of 2, 3, 4 and 5mm for wide platform and golden colour anodized.
 - Rotational (pink colour anodized) or non-rotational (golden colour anodized) slim straight aesthetic multi-position abutment with shoulder heights of 2, 3, 4, 5 and 6 mm.
 - Transformable to external aesthetic straight multi-position abutment (golden anodized) with shoulder height 2, 3 and 4mm
- External Connection
 - Non-rotational aesthetic straight multi-position abutment with shoulder heights of 2, 3 and 5 mm and golden colour anodized.
 - May be converted to rotational using the octagonal castable abutment.

The shoulder height will depend on the soft tissue height. Supplied both individually and as a kit with:

- 2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic).
- 1 transepithelial transfer attachment
- 1 screw for the transepithelial transfer attachment
- 1 transepithelial closing abutment

Recommended torque:

- 30 Ncm aesthetic straight multi-position abutment.
- 25 Ncm prosthetic screw.

Do not reuse. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS					
	Internal Connection				
	Height	Platform			
		Narrow			Wide
		MU: unit sale // KMU: kit sale			
Rotational	1mm	MUSR 04010 KMUSR 04010	-	-	-
	2mm	MUSR 04020 KMUSR 04020	-	MUSLAR 04020 KMUSLAR 04020	MUSR 05020 KMUSR 05020
	3mm	MUSR 04030 KMUSR 04030	-	MUSLAR 04030 KMUSLAR 04030	MUSR 05030 KMUSR 05030
	4mm	MUSR 04040 KMUSR 04040	-	MUSLAR 04040 KMUSLAR 04040	MUSR 05040 KMUSR 05040
	5mm	MUSR 04050 KMUSR 04050	-	MUSLAR 04050 KMUSLAR 04040	MUSR 05050 KMUSR 05050
	6mm	-	-	MUSLAR 04060 KMUSLAR 04060	-

Non-rotational	1mm	MUSA 04010 KMUSA 04010	MUSA S04010 KMUSA S04010	-	-
	2mm	MUSA 04020 KMUSA 04020	MUSA S04020 KMUSA S04020	MUSLA 04020 KMUSLA 04020	MUSA S05020 KMUSA S05020
	3mm	MUSA 04030 KMUSA 04030	MUSA S04030 KMUSA S04030	MUSLA 04030 KMUSLA 04030	MUSA S05030 KMUSA S05020
	4mm	MUSA 04040 KMUSA 04040	MUSA S04040 KMUSA S04040	MUSLA 04040 KMUSLA 04040	MUSA S05040 KMUSA S05020
	5mm	MUSA 04050 KMUSA 04050	MUSA S04050 KMUSA S04050	MUSLA 04050 KMUSLA 04050	MUSA S05050 KMUSA S05020
	6mm	-	-	MUSLA 04060 KMUSLA 04060	-

Transformable to External	2mm	MUSTE 04020 KMUSTE 04020		-
	3mm	MUSTE 04030 KMUSTE 04030		-
	4mm	MUSTE 04040 KMUSTE 04040		-

Compatible Imlant Model	
ICI IPX	

CONFIGURATIONS				
Platform	Altura	External Connection		
		Rotational		Non-rotational
		Narrow		
		MU: unit sale // KMU: kit sale		
Narrow	2mm	MUSR 4020 / KMUSR 4020		MUSA 4020 / KMUSA 4020
	3mm	MUSR 4030 / KMUSR 4030		MUSA 4030 / KMUSA 4030
	4mm	-		-
	5mm	MUSR 4050 / KMUSR 4050		MUSA 4050 / KMUSA 4050
Compatible Implant Model				
ICE IPXE				

Angled Aesthetic Multi-position Abutments

Intended use

Our angled aesthetic multi-position abutments with direct implant connection are designed for multi-unit screw-retained restorations, with the aim of restoring the patient's mastication function and phonetics.

Indications

- Our angled aesthetic multi-position abutments are recommended in the following cases:
- Fixed multi-unit screw-retained restorations.
 - Restorations of the posterior sector.
 - Correction of implants with large non-parallelisms.

Description

Galimplant® angled aesthetic multi-position abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (external and internal), platforms, diameters and lengths. Our angled aesthetic multi-position abutments are available in the following configurations:

- Rotational Internal Connection:
 - 15° angled aesthetic multi-position abutment with shoulder heights of 3, 4 and 5mm and pink colour anodized.
 - 30° angled aesthetic multi-position abutment with shoulder heights of 3, 4 and 5mm and pink colour anodized.
 - 45° angled aesthetic multi-position abutment with shoulder height of 5mm and pink colour anodized.
 - 50° angled aesthetic multi-position abutment with shoulder height of 5mm and pink colour anodized.
 - 60° angled aesthetic multi-position abutment with shoulder height of 5mm and pink colour anodized.

The shoulder height will depend on the soft tissue height and the angulation will depend on the degree of the implant's inclination. Supplied both individually and as a kit with:

- 1 transporter
- 1 rotational castable abutment

-2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic)
-1 transepithelial transfer attachment
-1 screw for the transepithelial transfer attachment
-1 transepithelial closing abutment
Recommended torque:
-30 Ncm screw to fix the abutment to the implant
-10 Ncm prosthetic screw to fix the abutment to the prosthesis
Do not reuse. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS		Internal Connection
		Rotational
Height	Angulation	EMU: unit sale // KEMU: kit sale
3mm	15°	EMU 1504030 / KEMU 1504030
	30°	EMU 3004030 / KEMU 3004030
4mm	15°	EMU 1504040 / KEMU 1504040
	30°	EMU 3004040 / KEMU 3004040
5mm	15°	EMU 1504050 / KEMU 1504050
	30°	EMU 3004050 / KEMU 3004050
	45°	EMU 4504050 / KEMU 4504050
	50°	EMU 5004050 / KEMU 5004050
	60°	EMU 6004050 / KEMU 6004050
Compatible Implant Model		ICI IPX

Immediate Loading Abutments

Intended use

Our immediate loading abutments with direct implant connection are intended to support provisional single- or multiple-unit restorations in both cement- and screw-retained restorations with the aim of restoring the patient's mastication function and phonetics.

Indications

Our immediate loading abutments are recommended in the following cases:

- Provisional single- or multi-unit restorations.
- Provisional cement- or screw-retained restorations.

Description

Galimplant® immediate loading abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (internal and external), platforms, diameters and lengths.

Our immediate loading abutments are available in the following configurations:

- Internal connection:
 - Rotational immediate loading abutments and golden colour anodized.
 - Non-rotational immediate loading abutments with height of 1, 2, 3 y 4 mm and golden colour anodized.
- External connection:
 - Rotational immediate loading abutments and golden colour anodized.
 - Non-rotational immediate loading abutments and golden colour anodized.

Supplied both individually and as a kit with 2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic).

Recommended torque 30 Ncm.

Do not re-use. Single use.

Precautions during use

- Tissue reaction caused by provisional cement.
- Difficulties removing remaining cement.
- Less control of the crown or bridge's settling during the cementing process.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS					
External Connection			Internal Connection		
P: unit sale // EP: kit sale					
Height	Rotational	Non-rotational	Height	Rotational	Non-rotational
1mm	PCIR 40 EPCIR 40	PCIA 40 EPCIA 40	1mm	PCIR 040 EPCIR 040	PCIA 04010 EPCIA 04010
2mm	-	-	2mm	-	PCIA 04020 EPCIA 04020
3mm	-	-	3mm	-	PCIA 04030 EPCIA 04030
4mm			4mm	-	PCIA 04040 EPCIA 04040
Compatible Implant Model					
ICE IPXE			ICI IPX		

Immediate Loading Abutments for Multi-position Abutment

Intended use

The immediate loading abutments for multi-position abutments are intended to support provisional single- or multiple-unit restorations in both cement- and screw-retained restorations with the aim of restoring the patient's mastication function and phonetics.

Indications

The immediate loading abutments for multi-position abutments are recommended in the following cases:

When immediate loading abutment is used with multi-position straight abutment:

- Provisional unit restorations and multiple screwed.

When immediate loading abutment is used with multi-position angled abutment:

- Provisional multiple screwed restorations.

Description

The immediate loading abutments for galimplant® multi-position abutments are made from grade 5 titanium and can be used in galimplant® multi-position straight and angled abutments with different connection types (internal and external) and height.

The immediate loading abutments for galimplant® multi-position abutments are available in the following configurations: rotational (pink colour anodized) and non-rotational (golden colour anodized).

Supplied both individually and as a kit with 2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic).

Recommended torque 30 Ncm.

Do not re-use. Single use.

Precautions during use

- Tissue reaction caused by provisional cement.
- Difficulties removing remaining cement.
- Less control of the crown or bridge's settling during the cementing process.

See the following table for the prosthetic abutment models compatible with each of the configurations.

CONFIGURATIONS					
Rotational			Non-rotational		
PTI: unit sale // EPTI: kit sale					
PTIMUTR 40 EPTIMUTR 40			PTIMUTA 40 EPTIMUTA 40		
Compatible Multi-position Abutments Models					
#	Straight Multi-position		Angled Multi-position		Angled Aesthetic Multi-position
	MUSTCE	MUSTCI	MUCE	MUCI	EMUCI

Immediate Loading Abutments for Aesthetic Straight Multi-position Abutment

Intended use

The immediate loading abutments for aesthetic straight multi-position abutments are intended to support provisional single- or multiple-unit restorations in both cement- and screw-retained restorations with the aim of restoring the patient's mastication function and phonetics.

Indications

The immediate loading abutments for aesthetic straight multi-position abutments are recommended in the following cases:

- Provisional unit restorations and multiple screwed.

Description

The immediate loading abutments for galimplant® aesthetic straight multi-position abutments are made from grade 5 titanium and can be used in galimplant® multi-position straight aesthetic abutments with different connection types (internal and external), platform and height.

The immediate loading abutments for galimplant® aesthetic straight multi-position abutments are available in the following configurations rotational (pink colour anodized) and non-rotational (golden colour anodized).

Supplied both individually and as a kit with 2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic).

Recommended torque 30 Ncm.

Do not re-use. Single use.

See the following table for the prosthetic abutment models models compatible with each of the configurations.

CONFIGURATIONS			
PTI: unit sale // EPTI: kit sale			
Rotational		Non-rotational	
Narrow	Wide	Narrow	Wide
PTIMUR 4048 EPTIMUR 4048	PTIMUR 5048 EPTIMUR 5048	PTIMUA 4048 EPTIMUA 5048	PTIMUA 5048 EPTIMUA 5048
Compatible Multi-position Abutments Models			
Aesthetic Straight Multi-position		Aesthetic Straight Multi-position	
#	MUSCE MUSCI	#	MUSCE MUSCI
Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.	Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.
Rotational slim aesthetic straight multi-position abutment.		Rotational slim aesthetic straight multi-position abutment.	

Interfaces for Multi-position Abutments

Intended use

The interfaces for multi-position abutments are designed for use with multi-position straight and angled abutments with the aim of restoring the patient's mastication function and phonetics.

Indications

The interfaces for multi-position abutments are recommended for multiple cemented-screwed rehabilitations.

Description

The galimplant® interfaces for multi-position abutments are made from grade 5 titanium and can be used in galimplant® multi-position straight and angled abutment with different connection types (internal and external) and height.

The galimplant® interfaces for multi-position abutments are available in rotational configuration and pink colour anodized.

Supplied both individually and as a kit with 2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic).

Recommended torque 10 Ncm.

Do not re-use. Single use.

Precautions during use

- Tissue reaction caused by provisional cement.
- Difficulties removing remaining cement.
- Less control of the crown or bridge's settling during the cementing process.

See the following table for the prosthetic abutment models compatible with each of the configurations.

CONFIGURATIONS					
Rotational		Non-rotational			
PITE: unit sale // EPITE: kit sale					
PITEMUTR 4040 EPITEMUTR 4040		-			
Compatible Multi-position Abutments Models					
#	Straight Multi-position		Angled Multi-position		Angled Aesthetic Multi-position
	MUSTCE	MUSTCI	MUCE	MUCI	EMUCI

Interfaces for Multi-position Straight Aesthetic Abutments

Intended use

The interfaces for aesthetic straight multi-position abutments are designed for use with multi-position straight aesthetic abutments with the aim of restoring the patient's mastication function and phonetics.

Indications

The interfaces for aesthetic straight multi-position abutments are recommended for unitary or multiple cemented-screwed rehabilitations.

Description

The galimplant® interfaces for multi-position straight aesthetic abutments are made from grade 5 titanium and can be used in galimplant® multi-position straight aesthetic abutment with different connection types (internal and external), platform and height.

The interfaces for galimplant® aesthetic straight multi-position abutments are available with narrow (3.8mm) and wide (4.8mm) platforms, both with rotational (pink anodized) and anti-rotational (gold anodized) configurations and heights 4, 7 and 7.5mm, the latter configuration being cut to 6 and 4.4mm.

Supplied with 2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic).

Recommended torque 25 Ncm.

Do not re-use. Single use.

Precautions during use

- Tissue reaction caused by provisional cement.
- Difficulties removing remaining cement.
- Less control of the crown or bridge's settling during the cementing process.

See the following table for the prosthetic abutment models compatible with each of the configurations.

CONFIGURATIONS				
Rotational		Non-rotational		Transformable to External
Narrow	Wide	Narrow	Wide	
EPITEMUR 4040 EPITEMUR 4070 EPITEMUR DA4040 EPITEMUR DA4075	EPITEMUR 5040 EPITEMUR 5070 EPITEMUR DA5040 EPITEMUR DA5075	EPITEMUA 4040 EPITEMUA 4070 EPITEMUA DA4040 EPITEMUA DA4075	EPITEMUA 5040 EPITEMUA 5070 EPITEMUA DA5040 EPITEMUA DA5075	EPITEMUA 40T EPITEMUA 70T

Compatible Multi-position Abutments Models				
Aesthetic Straight Multi-position		Aesthetic Straight Multi-position		
#	MUSCE MUSCI	#	MUSCE MUSCI	
Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.	Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.	Transformable to external rotational aesthetic straight multi-position abutment
Rotational slim aesthetic straight multi-position abutment.		Rotational slim aesthetic straight multi-position abutment.		

Screw-retained Prosthesis Base

Intended use

Our screw-retained prosthesis bases with direct implant connection over which the crown is cemented in the laboratory, are designed for definitive screw-retained restorations with the aim of restoring the patient's mastication function and phonetics.

Indications

The use of screw-retained prosthesis bases is recommended in fixed single- or multi-unit screw-retained restorations.

Description

The galimplant® screw-retained prosthesis base is made from grade 5 titanium and can be used in galimplant® implants with different connection types (internal and external), platforms, heights and lengths.

Our galimplant® screw-retained prosthesis bases are available in the following configurations:

- Internal Connection:
 - Rotational screw-retained prosthesis bases and pink colour anodized.
 - Non-rotational screw-retained prosthesis bases and golden colour anodized.
- External Connection:
 - Non-rotational screw-retained prosthesis bases and golden colour anodized.

Supplied with 2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic).

Recommended torque 30 Ncm.

Do not re-use. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS		External Connection	Internal Connection		
Platform	Height	Non-rotational	Rotational	Non-rotational	
Narrow	1mm	EPGZA 40	EPGZR 04010	EPGZA 04010	
	2mm	-	EPGZR 04020	EPGZA 04020	EPGZA 040207
	3mm	-	EPGZR 04030	EPGZA 04030	EPGZA 040307
	4mm	-	-	EPGZA 04040	EPGZA 040407
	5mm	-	-	EPGZA 04050	EPGZA 040507
Wide	2mm	-	-	EPGZA 05020	EPGZA 050207
	3mm	-	-	EPGZA 05030	EPGZA 050307
	4mm	-	-	EPGZA 05040	EPGZA 050407
	5mm	-	-	EPGZA 05050	EPGZA 050507
Compatible Implant Model		ICE IPXE	ICI IPX		

Screw-retained Prosthesis Base for Aesthetic Straight Multi-position Abutments

Intended use

Our screw-retained prosthesis bases for galimplant® aesthetic straight multi-position abutments with direct implant connection over which the crown is cemented in the laboratory, are designed for definitive screw-retained restorations with the aim of restoring the patient's mastication function and phonetics.

Indications

The use of screw-retained prosthesis bases for galimplant® aesthetic straight multi-position abutments is recommended in fixed single- or multi-unit screw-retained restorations.

Description

The galimplant® screw-retained prosthesis base for galimplant® aesthetic straight multi-position abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (internal and external), platforms, diameters and lengths.

Our galimplant® screw-retained prosthesis bases for galimplant® aesthetic straight multi-position abutments are available for rotational (pink colour anodized) or not-rotational (golden colour anodized) configurations.

Supplied with 2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic).

Recommended torque 30 Ncm.

Do not re-use. Single use.

See the following table for the prosthetic abutment models compatible with each of the configurations.

CONFIGURATIONS				
Rotational		Non-rotational		Transformable to External
Narrow	Wide	Narrow	Wide	
EPGZMUR 4040 EPGZMUR 4070	EPGZMUR 5040 EPGZMUR 5070	EPGZMUA 4040 EPGZMUA 4040	EPGZMUA 4040 EPGZMUA 4040	EPGZA 40T EPGZA 70T

Compatible Multi-position Abutments Models				
Aesthetic Straight Multi-position Abutment				
#	MUSCE MUSCI			
Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.	Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.	Transformable to external rotational aesthetic straight multi-position abutment
Rotational slim aesthetic straight multi-position abutment.		Rotational slim aesthetic straight multi-position abutment.		

Compatible Interfaces with Cerec System

Intended use

The compatible interfaces with Cerec System are intended for use with Cerec system with the aim of restoring the patient's mastication function and phonetics.

Indications

The compatible interfaces with Cerec System allow the realization of unitary cemented-screwed restorations achieving an optimal fit with the implant.

Description

The compatible interfaces with Cerec System are made from grade 5 titanium and can be used in galimplant® internal connection implants, with different platforms, diameters and lengths.

The compatible interfaces with Cerec System are available in golden colour anodized and Internal Connection for heights of 1, 2, 3 and 4 mm.

Supplied with 2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic).

Recommended torque 30 Ncm.

Do not re-use. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS	
Height	Internal Connection
1mm	EPCERC 040
2mm	EPCERC 04020
3mm	EPCERC 04030
4mm	EPCERC 04040
Compatible Implant Model	
ICI IPX	

Compatible Interfaces with Cerec System for Aesthetic Straight Multi-position Abutments

Intended use

The compatible interfaces with Cerec System for galimplant® aesthetic straight multi-position abutments are intended for use with Cerec system with the aim of restoring the patient's mastication function and phonetics.

Indications

The compatible interfaces with Cerec System for galimplant® aesthetic straight multi-position abutments allow the realization of unitary cemented-screwed restorations achieving an optimal fit with the implant.

Description

The compatible interfaces with Cerec System for galimplant® aesthetic straight multi-position abutments are made from grade 5 titanium and can be used in galimplant® internal connection implants, with different platforms, diameters and lengths.

The compatible interfaces with Cerec System for galimplant® aesthetic straight multi-position abutments are available in rotational (pink anodized) or anti-rotational (gold anodized) configurations.

Supplied with 2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic).

Recommended torque 30 Ncm.

Do not re-use. Single use.

See the following table for the prosthetic abutment models compatible with each of the configurations.

CONFIGURATIONS			
Rotational		Non-rotational	
Narrow	Wide	Narrow	Wide
EPCERC MUR 40	EPCERC MUR 50	EPCERC MUA 40	EPCERC MUA 50
Compatible Multi-position Abutments Models			
Aesthetic Straight Multi-position Abutment			
#	MUSCE MUSCI		
Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.	Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.
Rotational slim aesthetic straight multi-position abutment.		Rotational slim aesthetic straight multi-position abutment.	

Cast-to Abutments

Intended use

Our cast-to abutments with a machined base are designed for definitive screw-retained restorations with the aim of restoring the patient's mastication function and phonetics.

Indications

Our cast-to abutments are recommended in single or multi-unit restorations when a cast of the most coronal part is necessary.

Description

Galimplant® cast-to abutments can be used in galimplant® implants with different connection types (internal and external), platforms, diameters and lengths. Our cast-to abutments are available in the following configurations: rotational and non-rotational. **They are to be used specifically for the elaboration of dental crowns in the prosthetic laboratory.** Supplied with 2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic). The recommended torque for the prosthetic screw is 30 Ncm. Do not re-use. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS			
External Connection		Internal Connection	
Rotational	Non-rotational	Rotational	Non-rotational
EPSR 40	EPSA 40	EPSR 040	EPSA 040
Compatible Implant Model			
ICE IPXE		ICI IPX	

Cast-to Abutment for Multi-position Straight Aesthetic

Intended use

Our cast-to abutments for galimplant multi-position straight aesthetic abutments with a machined base are designed for definitive screw-retained restorations with the aim of restoring the patient's mastication function and phonetics.

Indications

Our cast-to abutments for galimplant multi-position straight aesthetic abutments are recommended in single or multi-unit restorations when a cast of the most coronal part is necessary.

Description

The cast-to abutment for galimplant® multi-position straight aesthetic abutments can be used in galimplant® multi-position straight aesthetic original abutments with different connection, platform and height types. Our cast-to abutments for galimplant multi-position straight aesthetic abutments are available in the following configurations: rotational and non-rotational. **They are to be used specifically for the elaboration of dental crowns in the prosthetic laboratory.** Supplied with 2 screws blue colour anodized (one clinical, packed inside a capsule and another prosthetic). The recommended torque for the prosthetic screw is 30 Ncm. Do not re-use. Single use.

CONFIGURATIONS			
Rotational		Non-rotational	
Narrow	Wide	Narrow	Wide
EPSMUR 4048	EPSMUR 5048	EPSMUA 4048	EPSMUA 5048
Compatible Multi-position Abutments Models			
Aesthetic Straight Multi-position Abutment			
#	MUSCE MUSCI		
Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.	Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.
Rotational slim aesthetic straight multi-position abutment.		Rotational slim aesthetic straight multi-position abutment.	

Overdent Abutments

Intended use

The overdent abutments with direct implant connection are intended to retain and support the detachable prosthesis, with the aim of restoring the patient's mastication function and phonetics.

Indications

The use of overdent abutments is recommended in the following cases:

- Supported-implant prostheses
- Retained-implant prostheses

They enable the correction of the implants' angulation of up to 10 degrees.

Description

Galimplant® overdent abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (internal and external), platforms, diameters and lengths. Our overdent abutments are available for internal and external connections with shoulder heights of 0.8, 1.6, 3, 4, 5 and 7 mm. The shoulder height will depend on the soft tissue height. In the case of divergent implants, we can resolve non-parallelism up to an angulation of 10° with this abutment. Supplied with:

- 1 cap
- 4 retainers

Recommended torque 30 Ncm. Only the galimplant® overdent wrench should be used. Do not re-use. Single use.

To choose the height of the overdent abutment

- Measure the height of the tissue from the implant platform up to the highest point of the gingival crest (or tissue) surrounding the implant.
- Choose the corresponding abutment which has exactly the same measurement as the tissue, or the next highest measurement, never lower.
- The overdent abutment has a reference mark around the whole abutment which should be on the same level as the gum.

Recommendations

- Periodically replace the retainers. To ensure proper maintenance of the prosthesis, the clinician must review the retention of the prosthesis every 6 months and replace the retainers if necessary.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS		
Height	External Connection	Internal Connection
0,8 mm	KO 4008	KO 04008
1,6 mm	KO 4016	KO 04016
3 mm	KO 4030	KO 04030
4 mm	KO 4040	KO 04040
5 mm	KO 4050	KO 04050
7 mm	KO 4070	KO 04070
Compatible Implant Model	ICE IPXE	ICI IPX

Ball Abutments

Intended use

Our ball abutments with direct implant connection are intended for retained-implant or supported-implant prostheses with the aim of restoring the patient's mastication function and phonetics.

Indications

The use of ball abutments is recommended in the following cases:

- Supported-implant prostheses
- Retained-implant prostheses

They enable an angulation correction of up to 30 degrees.

Description

Galimplant® ball abutments are made from grade 5 titanium and can be used in galimplant® internal connection implants with different platforms, diameters and lengths. Our ball abutments are available with the following shoulder heights: 0.8, 1.6, 3, 4 and 5 mm. Their height will depend on the soft tissue height. In the case of divergent implants, with this abutment we can resolve non-parallelism up to an angulation of 30°. Supplied with:

- 1 cap
- 1 O-ring

Recommended torque 30 Ncm. Do not re-use. Single use.

Recommendations

- Periodically replace the retaining bond.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS	
Height	Internal Connection
0,8 mm	KB 04008
1,6 mm	KB 04016
3 mm	KB 04030
4 mm	KB 04040
5 mm	KB 04050
Compatible Implant Model	IPX ICI

Contraindications

The use of prosthetic abutments is not recommended for patients:

- who do not meet the necessary medical requirements for dental implant surgery.
- who are allergic or hypersensitive to commercially pure titanium or Ti-6Al-4V titanium alloy (titanium, aluminium and vanadium).

Side effects, interactions and complications

The side effects, interactions and complications are those derived from implant placement procedures.

Possible complications following prosthetic restoration include:

- Complications: Loosening/fracture of the abutment and/or screw, mechanical problems associated with the prosthesis, loss of prosthesis retention, loss of the hole sealing of the screw-retained prosthesis, microgap in the implant-abutment connection and microleakage.

If one of these disorders/complications appears, the patient should consult their specialist doctor who will give them the corresponding instructions.

Storage and transport

The prosthetic abutments are delivered packed in peel-pack bags inside a closed cardboard blister on which the label is placed and includes all product information, including the date of manufacture, which can be deduced from the batch number, the first four digits corresponding to the year and the next two to the month of manufacture (YYYYMMXXX). This packaging contains and protects the product during storage and transport and does not have a specified shelf-life. Prosthetic abutments must be stored in a dry place in the original packaging at room temperature and not exposed to direct sunlight. When removing the product from its packaging one must verify that it is not damaged and that the product fits the description on the label. **Caution!** All our prosthetic abutments are sold in a NON STERILE state. Before use, they must be cleaned, disinfected and sterilised, observing the procedures validated by Nueva Galimplant SLU, included in the Cleaning/Disinfection/Sterilization section. Said procedures should be carried out before intraoral use. The repetition of the procedures described in this section do not alter the characteristics of these devices. The non-observance of these instructions may cause cross infections.

Cleaning/Sterilisation/Sterilization

The use of neutral or slightly alkaline cleaning agents is recommended. It is essential to verify that the cleaning product used is compatible with the medical devices are going to processed and respect the specifications of use (temperature, concentration and time) indicated by the manufacturer.

Cleaning and disinfection

To clean and disinfect the attachments either of the two following methods may be used:

Manual Method

- Place the attachments in a perforated tray or basket.
- Prepare an ultrasonic bath and a detergent and disinfectant solution in accordance with the manufacturer's specifications of the product used. *Nueva Galimplant S.L.U. has validated this process using the product neodisher® septo PreClean diluted at 1% to prepare the ultrasonic bath.*
- Immerse the basket with the attachments in the ultrasonic bath and keep them at a temperature of 20° C.
- Remove the attachments from the ultrasonic bath and rinse with plenty of purified water.
- If necessary, repeat the cleaning and disinfection step.

Automatic Method

- Place the attachments in the thermal washer-disinfector's tray or basket. Nueva Galimplant S.L.U. recommends thermal washer-disinfectors that comply with the EN ISO 15883 normative.
- Run the cleaning, disinfecting and drying cycle. *Nueva Galimplant S.L.U. has validated the Vario-TD program by the professional brand Miele, using the detergent neodisher® MediClean forte diluted at 0.5%. This program consists of:*
 - Cold water soak for 4 minutes.
 - 55° C clean for 10 minutes with detergent.
 - Cold water neutralisation for 6 minutes.
 - Cold water mid-cycle rinse for 3 minutes.
 - 93° C thermal disinfection for 5 minutes.
 - 110° C dry for 25 minutes.

To guarantee optimum protection of the attachments we recommend that purified water is used throughout the whole process. Note: it is important to always follow the instructions provided by the detergent and thermal washer-disinfector manufacturers.

Packaging

Once they are clean, disinfected and completely dry, the attachments must be packaged in a sterilisation bag.

The packaging of thermally sterilised medical devices must comply with the following requisites:

- Be appropriate for steam sterilisation, that is, have a minimum thermic resistance of 137° C and be sufficiently permeable for steam.
- Protect the devices sufficiently from mechanical damage.
- Comply with the EN ISO 11607 normative.

Sterilisation

- Load the autoclave with the conditioned attachments in the sterilisation bags. Do not overload the autoclave and always follow the equipment manufacturer's instructions.
- Just like in the other steps of these instructions, we recommend the use of purified water to guarantee optimum quality.
- Nueva Galimplant S.L.U. recommends using an autoclave which complies with the requisites of the EN ISO 17665 normative, in addition to being validated and having adequate maintenance. The validation of the sterilisation was carried out in a steam autoclave using a 134° C pre-vacuum cycle for 6 minutes.

- To control the sterilisation process, the use of chemical indicators is recommended, such as, for example, the verification strips which the sterilisation bags come equipped with and which change colour when the sterilisation process has finished.

Note: it is important to always follow the autoclave manufacturer's instructions and keep maintenance up to date.

Caution! The re-use and/or re-sterilisation of prosthetic abutments can cause risk of infection and/or cross contamination.

Documentation, traceability and implant card.

Nueva Galimplant S.L.U. recommends keeping an exhaustive clinical, radiological and photographic documentation. All products can be identified through their reference and batch code and / or UDI code. To maintain the mandatory traceability of these products, two adhesive labels are included, one for the medical record and the other for the patient, the latter must be attached to the implant card supplied with the products and which must be delivered to the patient fully completed once the surgery is over.

WARNINGS

Only original prosthetic abutments should be placed in the galimplant® implants. The prosthetic and clinical practitioner must verify that the materials and techniques to be used during prosthetic and clinical procedures are appropriate and compatible with the galimplant® prosthetic abutments and must follow the manufacturer's instructions for use of these materials. Before the prosthetic part, the physician should carry out an exhaustive patient study, in order to determine a correct planning of the case. For the placement of the prosthetic abutments, use only original wrenches of the brand galimplant®. Consult the compatible product models in the catalogue available at www.galimplant.com. There is a risk of aspiration or ingestion of the products when they are used, for this reason, they should only be used by professionals. The prosthetic abutments must solely and exclusively be placed during the implants osteointegration or if the implants have a great primary stability. Failure to observe the procedures set out in these instructions could harm the patient and give rise to one or all of the following complications:

- Aspiration or swallowing of the components.
- Infection.
- Damage to the implant, abutment, components or instruments.
- Implant failure.

To guarantee a good long term treatment result, the clinician must inform the patient about appropriate home oral hygiene and the importance of attending periodic clinical and radiographic consultations to perform a control and preventive diagnosis of any side effect and/or complication.

In the event of a serious incident related to the use or result of use of the product, inform Nueva Galimplant (www.galimplant.com) and your National Competent Authority.

The practitioner must provided the patient with the information about side effects, interactions and complications in relation to galimplant® prosthetic abutments.

Procedures for the disposal of the product

Once they have been extracted from the oral cavity, the prosthetic components must be disposed of as biological waste, in accordance with current local laws.

Summary of Safety and Clinical Performance

The Summary of Safety and Clinical Performance (SSCP) for the product is available in the European database EUDAMED. The document can be obtained from the website: <https://ec.europa.eu/tools/eudamed>.

Validity

These instructions for use will be valid as of February 2025.

SYMBOLS

Warning, read the instructions for use

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Do not use if packaging has been damaged

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