Español

## Pilares Protésicos Galimplant

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**IMPORTANT! Read the information contained in these operating instructions carefully before using the product.**

**NUEVA GALIMPLANT** complies with all the regulations established by the European laws and directives relating to the manufacture and distribution of medical products. The galimplant® implant system, attachments and instruments are certified and authorised for sale by a Notified Body, guaranteeing the perfect quality of all its products. The use and application of the galimplant® system is not under the manufacturer's control, and the user is responsible for any damages which may occur due to an incorrect manipulation or use of the product.

#### Safety, responsibility and guarantee

We guarantee the quality of our products within the framework of our sales and supply conditions. This product is part of a global concept and should only be used in conjunction with the associated original products according to the Nueva Galimplant instructions and recommendations. The non-recommended use of products manufactured by third parties together with galimplant® products will void any guarantee or other obligation, express or implicit, by Nueva Galimplant, S.L.U.

The user has sole responsibility for the procedure and use of the product. It is the practitioner's responsibility to use the product in accordance with the current instructions for use and to determine if it is appropriate for each patient's individual situation. In particular, this responsibility is expressly applicable in the case of its use in non-recommended procedures. If in doubt, the user should consult us.

#### Assistance and training

Technical assistance concerning the use of our products will be carried out verbally, by written means, by electronic means or by demonstrations.

Said assistance will be given according to our knowledge of the current state of science and techniques when the product is on sale. This does not exempt the user from their obligation to personally make sure that the product is appropriate for the purposes and procedures planned. Nueva Galimplant, with the aim of ensuring the correct use of its products, offers courses and practical sessions about its products through the platform: galimplant.training.

#### Precautions and warnings

In general terms, our products must be held to avoid their aspiration during intraoral use or insertion in the implant.

Galimplant® prosthetic abutments have not been sterilised before delivery, so they must be cleaned, disinfected and sterilised before use and it is essential that they are handled in an aseptic manner.

Galimplant® prosthetic abutments are of single use. The reusing of these types of products could lead to a deterioration of their characteristics which involves risk of tissue infection, surgical or prosthodontic failure and/or damage to the patient's health.

The following descriptions are insufficient for the immediate application of any abutment from the galimplant® system, this system should only be used by dentists, doctors and surgeons. The dentist/doctor/surgeon must have knowledge in oral implantology and experience in the manipulation of galimplant® products, in order to use them safely and correctly, following the indications of these instructions for use.

#### Patient Groups and Clinical Benefits

The galimplant® prosthetic abutments with direct implant connection are intended for use in totally or partially edentulous people or with poor prognosis or non-restorable teeth that request treatment and that meet the criteria for psychological, medical, anatomical, surgical, prosthodontic and maintenance selection specified in these instructions for use. The clinical benefit of treatment is the replacement of the patient's missing teeth.

## Straight Abutments

#### Intended use

Our straight abutments with direct implant connection are solely and exclusively intended to help support cement-restorations with the aim of restoring the patient's mastication function and phonetics.

#### Indications

Our straight abutments are recommended in the following cases:

- Single or multi-unit restorations cemented on top of the abutment.

#### Description

Galimplant® straight abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (internal and external), platforms, diameters and lengths. Our straight abutments are available in the following configurations:

- Non-rotational straight abutment with shoulder heights of 0, 1, 3 and 5mm and golden colour anodized.
- Rotational straight abutment with shoulder heights of 0 mm and golden colour anodized. The shoulder height will depend on the soft tissue height.

Supplied both individually and as a kit with 2 screws blue colour anodized (one clinical and another prosthetic).

Recommended torque 30 Ncm.

Do not re-use. Single use.

#### Precautions during use

- Tissue reaction caused by provisional cement.
- Difficulties removing remaining cement.
- Less control of the crown or bridge's settling during the cementing process.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS		External connection	Internal connection
<i>P: unit sale // EP: kit sale</i>			
Rotational	0mm	PR 4000 EPR 4000	PR 04000 EPR 04000
	0mm	PA 4000 / EPA 4000	PA 04000 / EPA 04000
	1mm	PA 4010 / EPA 4010	PA 04010 / EPA 04010
Non-rotational	3mm	PA 4030 / EPA 4030	PA 04030 / EPA 04030
	5mm	PA 4050 / EPA 4050	PA 04050 / EPA 04050
	<b>Compatible Implant Model</b>	<b>ICE IPXE</b>	<b>ICI IPX</b>

## Angled Abutments

#### Intended use

Our angled abutments with direct implant connection are intended to resolve implant angulation problems, helping to support cement-restorations with the aim of restoring the patient's mastication function and phonetics.

#### Indications

The angled abutments are recommended in the following cases:

- Single- or multiple-unit cement-restorations.
- Fixed restorations with non-parallelism between implants.

#### Description

Galimplant® angled abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (internal and external), platforms, diameters and lengths.

Galimplant® angled abutments are available in the following configurations:

- 15° with shoulder heights of 1, 3 and 5 mm and golden colour anodized.
- 30° with shoulder heights of 1, 3 and 5 mm and golden colour anodized.

The amount of degrees will depend on the implant's inclination and the shoulder height will depend on the soft tissue height.

Supplied both individually and as a kit with 2 screws blue colour anodized (one clinical and another prosthetic).

Recommended torque 20-30 Ncm.

Do not re-use. Single use.

#### Precautions during use

- Tissue reaction caused by provisional cement.
- Difficulties removing remaining cement.
- Less control of the crown or bridge's settling during the cementing process.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS		External Connection	Internal Connection
<i>P: unit sale // EP: kit sale</i>			
Angulation	Height		
	1mm	PA 154010 / EPA 154010	PA 0154010 / EPA 0154010
	3mm	PA 154030 / EPA 154030	PA 0154030 / EPA 0154030
15°	5mm	PA 154050 / EPA 154050	PA 0154050 / EPA 0154050
	1mm	PA 304010 / EPA 304010	PA 0304010 / EPA 0304010
	3mm	PA 304030 / EPA 304030	PA 0304030 / EPA 0304030
30°	5mm	PA 304050 / EPA 304050	PA 0304050 / EPA 0304050
	<b>Compatible Implant Model</b>	<b>ICE IPXE</b>	<b>ICI IPX</b>

## Angulable Aesthetic Abutments

#### Intended use

Our angulable aesthetic abutments with direct implant connection are intended to resolve implant angulation problems, helping to support screw-retained restorations with the aim of restoring the patient's mastication function and phonetics.

#### Indications

The angulable aesthetic abutments are recommended in the following cases:

- Single- or multiple-unit screw-retained restorations.

#### Description

Galimplant® angulable aesthetic abutments are made from grade V titanium and can be used in galimplant® internal connection implants with different platforms, diameters and lengths.

Galimplant® angulable aesthetic abutments are available in the following configurations:

- Shoulder heights of 2, 3 and 4 mm and golden colour anodized.

The amount of the shoulder height will depend on the soft tissue height.

Supplied both individually and as a kit with 2 screws golden colour anodized (one clinical and another prosthetic).

Recommended torque 20-30 Ncm.

Do not re-use. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS	Internal Connection
Height	<i>PEA DP: unit sale // EPEA DP: kit sale</i>
2mm	PEA DP 04020 / EPEA DP 04020
3mm	PEA DP 04030 / EPEA DP 04030
4mm	PEA DP 04040 / EPEA DP 04040
<b>Compatible Implant Model</b>	<b>ICI IPX</b>

## Straight Multi-position Abutments

#### Intended use

Our straight multi-position abutments with direct implant connection are designed for single- or multi-unit screw-retained restorations, with the aim of restoring the patient's mastication function and phonetics.

#### Indications

Our multi-position straight abutments are recommended in fixed single- or multi-unit screw-retained restorations.

#### Description

Galimplant® multi-position straight abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (external and internal), platforms, diameters and lengths.

Our straight abutments are available in the following configurations:

- Internal Connection
  - Non-rotational multi-position straight abutment with shoulder heights of 0, 1, 2 and 3 mm and golden colour anodized.
  - Rotational multi-position straight abutment with shoulder heights of 0, 1, 2 and 3 mm and pink colour anodized.
- External Connection
  - Non-rotational multi-position straight abutment with shoulder heights of 0, 1, 2 and 3 mm and golden colour anodized.
  - Rotational multi-position straight abutment with shoulder heights of 0, 1, 2 and 3 mm and pink colour anodized.

The shoulder height will depend on the soft tissue height.

Supplied both individually and as a kit with:

-2 screws blue colour anodized (one clinical and another prosthetic)

-1 transepithelial transfer attachment

-1 screw for the transepithelial transfer attachment

-1 transepithelial closing abutment

Recommended torque:

-30 Ncm multi-position straight abutment.

-10 Ncm prosthetic screw.

Do not reuse. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURACIONES					
Conexión Externa			Conexión Interna		
<i>MUST: venta unitaria // KMUST: venta en kit</i>					
Altura	Rotacional	Anti-rotacional	Altura	Rotacional	Anti-rotacional
1mm	MUST 4010 KMUST 4010	MUSTA 4010 KMUSTA 4010	1mm	MUST 04010 KMUST 04010	MUSTA 04010 KMUSTA 04010
2mm	MUST 4020 KMUST 4020	MUSTA 4020 KMUSTA 4020	2mm	MUST 04020 KMUST 04020	MUSTA 04020 KMUSTA 04020
3mm	MUST 4030 KMUST 4030	MUSTA 4030 KMUSTA 4030	3mm	MUST 04030 KMUST 04030	MUSTA 04030 KMUSTA 04030
<b>Modelo Implante Compatible</b>					
<b>ICE IPXE</b>			<b>ICI IPX</b>		

## Multi-position Angled Abutments

#### Intended use

Our multi-position angled abutments with direct implant connection are designed for multi-unit screw-retained restorations, with the aim of restoring the patient's mastication function and phonetics.

#### Indications

Our multi-position angled abutments are recommended in the following cases:

- Fixed multi-unit screw-retained restorations.
- Restorations of the posterior sector.
- Correction of implants with large non-parallelisms.

#### Description

Galimplant® multi-position angled abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (external and internal), platforms, diameters and lengths.

Our multi-position angled abutments are available in the following configurations:

- Rotational Internal Connection:
  - Multi-position abutment angled at 15° with shoulder heights of 2 and 3 mm and pink colour anodized.
  - Multi-position abutment angled at 30° with shoulder height of 3 mm and pink colour anodized.
  - Multi-position abutment angled at 45° with shoulder height of 5 mm and pink colour anodized.
  - Multi-position abutment angled at 50° with shoulder height of 5 mm and pink colour anodized.
  - Multi-position abutment angled at 60° with shoulder height of 5 mm and pink colour anodized.
- Rotational External Connection:
  - Multi-position abutment angled at 15° with shoulder heights of 2 and 3 mm and pink colour anodized.
  - Multi-position abutment angled at 30° with shoulder height 3 and 4 mm and pink colour anodized.

The shoulder height will depend on the soft tissue height and the angulation will depend on the degree of the implant's inclination.

Supplied both individually and as a kit with:

-1 transporter

-2 screws blue colour anodized (one clinical and another prosthetic)

-1 transepithelial transfer attachment

-1 screw for the transepithelial transfer attachment

-1 transepithelial closing abutment

Recommended torque:

-30 Ncm screw to fix the abutment to the implant

-10 Ncm prosthetic screw to fix the abutment to the prosthesis

Do not reuse. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS		External Connection	Internal Connection
<i>MU: unit sale // KMU: kit sale</i>			
Height	Angulation	Rotational	Rotational
2mm	15°	MU 154020 / KMU 154020	MU 1504020 / KMU 1504020
3mm	15°	MU 154030 / KMU 154030	MU 1504030 / KMU 1504030
	30°	MU 304030 / KMU 304030	MU 3004030 / KMU 3004030
4mm	30°	MU 304040 / KMU 304040	MU 3004040 / KMU 3004040
	45°	-	MU 4504050 / KMU 4504050
	50°	-	MU 5004050 / KMU 5004050
5mm	60°	-	MU 6004050 / KMU 6004050

<b>Compatible Implant Model</b>	<b>ICE IPXE</b>	<b>ICI IPX</b>
---------------------------------	---------------------	--------------------

## Aesthetic Straight Multi-position Abutments

#### Intended use

Our aesthetic straight multi-position abutments with direct implant connection are designed for screw-retained restorations, with the aim of restoring the patient's mastication function and phonetics.

#### Indications

Our aesthetic straight multi-position abutments are recommended in fixed single- or multi-unit screw-retained restorations with high aesthetic requirements.

Our non-rotational multi-position slim straight aesthetic abutments are only indicated for single-unit prostheses on small teeth that support vertical loads.

Our rotational multi-position slim straight aesthetic abutments are only indicated for multiple-unit prostheses on small teeth that support vertical loads.

#### Description

Galimplant® aesthetic straight multi-position abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (external and internal), platforms, diameters and lengths.

Our straight aesthetic abutments are available in the following configurations:

- Internal Connection
  - Rotational aesthetic straight multi-position abutment with shoulder heights of 1, 2, 3, 4 and 5 mm for narrow platform and heights of 2, 3, 4 and 5mm for wide platform and pink colour anodized.
  - Non-rotational aesthetic straight multi-position abutment with shoulder heights of 1, 2, 3, 4 and 5 mm for narrow platform and golden colour anodized.
  - Solidary non-rotational aesthetic straight multi-position abutment with shoulder heights of 1, 2, 3, 4 and 5 mm for narrow platform and heights of 2, 3, 4 and 5mm for wide platform and golden colour anodized.
  - Rotational (pink colour anodized) or non-rotational (golden colour anodized) slim straight aesthetic multi-position abutment with shoulder heights of 2, 3, 4, 5 and 6 mm.
  - Transformable to external aesthetic straight multi-position abutment (pink colour anodized) with shoulder height 2, 3 and 4mm
- External Connection
  - Non-rotational aesthetic straight multi-position abutment with shoulder heights of 2, 3 and 5 mm and golden colour anodized.
  - May be converted to rotational using the octagonal castable abutment.

The shoulder height will depend on the soft tissue height.

Supplied both individually and as a kit with:

-2 screws blue colour anodized (one clinical and another prosthetic).

-1 transepithelial transfer attachment

-1 screw for the transepithelial transfer attachment

-1 transepithelial closing abutment

Recommended torque:

-30 Ncm aesthetic straight multi-position abutment.

-25 Ncm prosthetic screw.

Do not reuse. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS				
Height	Internal Connection			
	Platform			
	Narrow		Wide	
	<i>MUS: unit sale // KMUS: kit sale</i>			
Rotational	1mm	MUSR 04010 KMUSR 04010	-	-
	2mm	MUSR 04020 KMUSR 04020	-	MUSLAR 04020 KMUSLAR 04020
	3mm	MUSR 04030 KMUSR 04030	-	MUSLAR 04030 KMUSLAR 04030
	4mm	MUSR 04040 KMUSR 04040	-	MUSLAR 04040 KMUSLAR 04040
	5mm	MUSR 04050 KMUSR 04050	-	MUSLAR 04050 KMUSLAR 04050
	6mm	-	-	MUSLAR 04060 KMUSLAR 04060

Non-rotational	1mm	MUSA 04010 KMUSA 04010	MUSA S04010 KMUSA S04010	-	-
	2mm	MUSA 04020 KMUSA 04020	MUSA S04020 KMUSA S04020	MUSLA 04020 KMUSLA 04020	MUSA S05020 KMUSA S05020
	3mm	MUSA 04030 KMUSA 04030	MUSA S04030 KMUSA S04030	MUSLA 04030 KMUSLA 04030	MUSA S05030 KMUSA S05030
	4mm	MUSA 04040 KMUSA 04040	MUSA S04040 KMUSA S04040	MUSLA 04040 KMUSLA 04040	MUSA S05040 KMUSA S05020
	5mm	MUSA 04050 KMUSA 04050	MUSA S04050 KMUSA S04050	MUSLA 04050 KMUSLA 04050	MUSA S05050 KMUSA S05020
	6mm	-	-	MUSLA 04060 KMUSLA 04060	-

Transformable to External	1mm	MUSTE 04010 KMUSTE 04010	-	-
	2mm	MUSTE 04020 KMUSTE 04020	-	-
	3mm	MUSTE 04030 KMUSTE 04030	-	-
	4mm	MUSTE 04040 KMUSTE 04040	-	-

Compatible Implant Model	
	<b>ICI IPX</b>

CONFIGURATIONS				
Platform	Altura	External Connection		
		Rotational		Non-rotational
		Narrow		
		<i>MUS: unit sale // KMUS: kit sale</i>		

-30 Ncm aesthetic straight multi-position abutment metric 2.  
 -25 Ncm prosthetic screw.  
 Do not reuse. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS		
Internal Connection Metric 2		
Wide Platform		
Height	Rotational	Anti-rotational
	<i>MUS: unit sale // KMUS: kit sale</i>	
2mm	MUSR M2 05020 KMUSR M2 05020	MUSA M2 S05020 KMUSA M2 S05020
3mm	MUSR M2 05030 KMUSR M2 05030	MUSA M2 S05030 KMUSA M2 S05030
4mm	MUSR M2 05040 KMUSR M2 05040	MUSA M2 S05040 KMUSA M2 S05040
5mm	MUSR M2 05050 KMUSR M2 05050	MUSA M2 S05050 KMUSA M2 S05050
Convertible to External		
1mm	MUSTE M2 04010 KMUSTER M2 04010	MUSTE M2 04010 KMUSTE M2 04010
2mm	MUSTE M2 04020 KMUSTER M2 04020	MUSTE M2 04020 KMUSTE M2 04020
3mm	MUSTE M2 04030 KMUSTER M2 04030	MUSTE M2 04030 KMUSTE M2 04030
4mm	MUSTE M2 04040 KMUSTER M2 04040	MUSTE M2 04040 KMUSTE M2 04040
Compatible Implant Model		
	ICI M2 IPX M2	

## Angled Aesthetic Multi-position Abutments

### Intended use

Our angled aesthetic multi-position abutments with direct implant connection are designed for multi-unit screw-retained restorations, with the aim of restoring the patient's mastication function and phonetics.

### Indications

Our angled aesthetic multi-position abutments are recommended in the following cases:

- Fixed multi-unit screw-retained restorations.
- Restorations of the posterior sector.
- Correction of implants with large non-parallelisms.

### Description

Galimplant® angled aesthetic multi-position abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (external and internal), platforms, diameters and lengths.

Our angled aesthetic multi-position abutments are available in the following configurations:

- Rotational Internal Connection:
- 15° angled aesthetic multi-position abutment with shoulder heights of 3, 4 and 5mm and pink colour anodized.
- 30° angled aesthetic multi-position abutment with shoulder heights of 3, 4 and 5mm and pink colour anodized.
- 45° angled aesthetic multi-position abutment with shoulder height of 5mm and pink colour anodized.
- 50° angled aesthetic multi-position abutment with shoulder height of 5mm and pink colour anodized.
- 60° angled aesthetic multi-position abutment with shoulder height of 5mm and pink colour anodized.

The shoulder height will depend on the soft tissue height and the angulation will depend on the degree of the implant's inclination.

Supplied both individually and as a kit with:

- 1 transporter
- 1 rotational castable abutment
- 2 screws blue colour anodized (one clinical and another prosthetic)
- 1 transepithelial transfer attachment
- 1 screw for the transepithelial transfer attachment
- 1 transepithelial closing abutment

Recommended torque:

- 30 Ncm screw to fix the abutment to the implant
- 10 Ncm prosthetic screw to fix the abutment to the prosthesis

Do not reuse. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS		Internal Connection	
		Rotational	
Height	Angulation	<i>EMU: unit sale // KEMU: kit sale</i>	
3mm	15°	EMU 1504030 / KEMU 1504030	
	30°	EMU 3004030 / KEMU 3004030	
4mm	15°	EMU 1504040 / KEMU 1504040	
	30°	EMU 3004040 / KEMU 3004040	
5mm	15°	EMU 1504050 / KEMU 1504050	
	30°	EMU 3004050 / KEMU 3004050	
	45°	EMU 4504050 / KEMU 4504050	
	50°	EMU 5004050 / KEMU 5004050	
60°	EMU 6004050 / KEMU 6004050		
Compatible Implant Model		ICI IPX	ICMS ICMT

## Angled Aesthetic Multi-position Abutments, Metric 2

### Intended use

Our angled aesthetic multi-position abutments metric 2 with direct implant connection are designed for multi-unit screw-retained restorations, with the aim of restoring the patient's mastication function and phonetics.

### Indications

Our angled aesthetic multi-position abutments metric 2 are recommended in the following cases:

- Fixed multi-unit screw-retained restorations.
- Restorations of the posterior sector.
- Correction of implants with large non-parallelisms.

### Description

Galimplant® angled aesthetic multi-position abutments metric 2 are made from grade 5 titanium and can be used in galimplant® implants with different platforms, diameters and lengths.

Our angled aesthetic multi-position abutments metric 2 are available in the following configurations:

- Rotational Internal Connection:
- 15° angled aesthetic multi-position abutment metric 2 with shoulder height 5mm and pink colour anodized.
- 30° angled aesthetic multi-position metric 2 abutment with shoulder height 4.5mm and pink colour anodized.
- 45° angled aesthetic multi-position metric 2 abutment with shoulder height of 6mm and pink colour anodized.

The shoulder height will depend on the soft tissue height and the angulation will depend on the degree of the implant's inclination.

Supplied both individually and as a kit with:

- 1 transporter
- 2 screws blue colour anodized (one clinical and another prosthetic)
- 1 transepithelial transfer attachment
- 1 screw for the transepithelial transfer attachment
- 1 transepithelial closing abutment

Recommended torque:

- 30 Ncm screw to fix the abutment to the implant
- 10 Ncm prosthetic screw to fix the abutment to the prosthesis

Do not reuse. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS		Internal Connection	
		Rotational	
Angulation	Height	<i>EMU: unit sale // KEMU: kit sale</i>	
15°	5mm	EMU M2 155050 / KEMU M2 155050	
30°	5.5mm	EMU M2 305055 / KEMU M2 305055	
45°	6mm	EMU M2 455060 / KEMU M2 455060	
Compatible Implant Model		ICI M2 IPX M2	

## Immediate Loading Abutments

### Intended use

Our immediate loading abutments with direct implant connection are intended to support provisional single- or multiple-unit restorations in both cement- and screw-retained restorations with the aim of restoring the patient's mastication function and phonetics.

### Indications

Our immediate loading abutments are recommended in the following cases:

- Provisional single- or multi-unit restorations.
- Provisional cement- or screw-retained restorations.

### Description

Galimplant® immediate loading abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (internal and external), platforms, diameters and lengths.

Our immediate loading abutments are available in the following configurations:

- Internal connection:
  - Rotational immediate loading abutments and golden colour anodized.
  - Non-rotational immediate loading abutments with height of 1, 2, 3 y 4 mm and golden colour anodized.
- External connection:
  - Rotational immediate loading abutments and golden colour anodized.
  - Non-rotational immediate loading abutments and golden colour anodized.

Supplied both individually and as a kit with 2 screws blue colour anodized (one clinical and another prosthetic).

Recommended torque 30 Ncm.

Do not re-use. Single use.

### Precautions during use

- Tissue reaction caused by provisional cement.
- Difficulties removing remaining cement.
- Less control of the crown or bridge's settling during the cementing process.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS					
External Connection			Internal Connection		
<i>PCI: unit sale // EPCI: kit sale</i>					
Height	Rotational	Non-rotational	Height	Rotational	Non-rotational
1mm	PCIR 40 EPCIR 40	PCIA 40 EPCIA 40	1mm	PCIR 040 EPCIR 040	PCIA 04010 EPCIA 04010
2mm	-	-	2mm	-	PCIA 04020 EPCIA 04020
3mm	-	-	3mm	-	PCIA 04030 EPCIA 04030
4mm	-	-	4mm	-	PCIA 04040 EPCIA 04040
Compatible Implant Model					
	ICE IPXE			ICI IPX	

## Immediate Loading Abutments for Multi-position Abutment

### Intended use

The immediate loading abutments for multi-position abutments are intended to support provisional single- or multiple-unit restorations in both cement- and screw-retained restorations with the aim of restoring the patient's mastication function and phonetics.

### Indications

The immediate loading abutments for multi-position abutments are recommended in the following cases:

When immediate loading abutment is used with multi-position straight abutment:

- Provisional unit restorations and multiple screwed.

When immediate loading abutment is used with multi-position angled abutment:

- Provisional multiple screwed restorations.

### Description

The immediate loading abutments for galimplant® multi-position abutments are made from grade 5 titanium and can be used in galimplant® multi-position straight and angled abutments with different connection types (internal and external) and height.

The immediate loading abutments for galimplant® multi-position abutments are available in the following configurations: rotational (pink colour anodized) and non-rotational (golden colour anodized).

Supplied both individually and as a kit with 2 screws blue colour anodized (one clinical and another prosthetic).

Recommended torque 30 Ncm.

Do not re-use. Single use.

### Precautions during use

- Tissue reaction caused by provisional cement.
- Difficulties removing remaining cement.
- Less control of the crown or bridge's settling during the cementing process.

See the following table for the prosthetic abutment models compatible with each of the configurations.

CONFIGURATIONS					
Rotational			Non-rotational		
<i>PTIMUT: unit sale // EPTIMUT: kit sale</i>					
	PTIMUTR 40 EPTIMUTR 40			PTIMUTA 40 EPTIMUTA 40	
Compatible Multi-position Abutments Models					
#	Straight Multi-position		Angled Multi-position		Angled Aesthetic Multi-position
	MUSTCE	MUSTCI	MUCE	MUCI	EMUCI

## Immediate Loading Abutments for Aesthetic Straight Multi-position Abutment

### Intended use

The immediate loading abutments for aesthetic straight multi-position abutments are intended to support provisional single- or multiple-unit restorations in both cement- and screw-retained restorations with the aim of restoring the patient's mastication function and phonetics.

### Indications

The immediate loading abutments for aesthetic straight multi-position abutments are recommended in the following cases:

- Provisional unit restorations and multiple screwed.

### Description

The immediate loading abutments for galimplant® aesthetic straight multi-position abutments are made from grade 5 titanium and can be used in galimplant® multi-position straight aesthetic abutments with different connection types (internal and external), platform and height.

The immediate loading abutments for galimplant® aesthetic straight multi-position abutments are available in the following configurations rotational (pink colour anodized) and non-rotational (golden colour anodized).

Supplied both individually and as a kit with 2 screws blue colour anodized (one clinical and another prosthetic).

Recommended torque 30 Ncm.

Do not re-use. Single use.

See the following table for the prosthetic abutment models compatible with each of the configurations.

CONFIGURATIONS							
<i>PTIMU: unit sale // EPTIMU: kit sale</i>							
Rotational				Non-rotational			
Narrow		Wide		Narrow		Wide	
	PTIMUR 4048 EPTIMUR 4048		PTIMUR 5048 EPTIMUR 5048		PTIMUA 4048 EPTIMUA 4048		PTIMUA 5048 EPTIMUA 5048
Compatible Multi-position Abutments Models							
Aesthetic Straight Multi-position				Aesthetic Straight Multi-position			
#	MUSCE MUSCI			#	MUSCE MUSCI		
Rotational aesthetic straight multi-position abutment with narrow platform.			Rotational aesthetic straight multi-position abutment with wide platform.	Rotational aesthetic straight multi-position abutment with narrow platform.			Rotational aesthetic straight multi-position abutment with wide platform.
Rotational slim aesthetic straight multi-position abutment.			Rotational aesthetic straight multi-position abutment with wide platform.	Rotational slim aesthetic straight multi-position abutment.			Rotational aesthetic straight multi-position abutment with wide platform.

## Interfaces for Multi-position Abutments

### Intended use

The interfaces for multi-position abutments are designed for use with multi-position straight and angled abutments with the aim of restoring the patient's mastication function and phonetics.

### Indications

The interfaces for multi-position abutments are recommended for multiple cemented-screwed rehabilitations.

### Description

The galimplant® interfaces for multi-position abutments are made from grade 5 titanium and can be used in galimplant® multi-position straight and angled abutment with different connection types (internal and external) and height.

The galimplant® interfaces for multi-position abutments are available in rotational configuration and pink colour anodized.

Supplied both individually and as a kit with 2 screws blue colour anodized (one clinical and another prosthetic).

Recommended torque 10 Ncm.

Do not re-use. Single use.

### Precautions during use

- Tissue reaction caused by provisional cement.
- Difficulties removing remaining cement.
- Less control of the crown or bridge's settling during the cementing process.

See the following table for the prosthetic abutment models compatible with each of the configurations.

CONFIGURATIONS					
Rotational		Non-rotational			
<i>PITEMUT: unit sale // EPITEMUT: kit sale</i>					
	PITEMUTR 4040 / EPITEMUTR 4040 PITEMUTR DA4040 / EPITEMUTR DA4040 PITEMUTR DA4075 / EPITEMUTR DA4075			-	
Compatible Multi-position Abutments Models					
#	Straight Multi-position		Angled Multi-position		Angled Aesthetic Multi-position
	MUSTCE	MUSTCI	MUCE	MUCI	EMUCI

## Interfaces for Multi-position Straight Aesthetic Abutments

### Intended use

The interfaces for aesthetic straight multi-position abutments are designed for use with multi-position straight aesthetic abutments and multi-position angled aesthetic abutments metric 2 and with the aim of restoring the patient's mastication function and phonetics.

### Indications

The interfaces for aesthetic straight multi-position abutments are recommended for unitary or multiple cemented-screwed rehabilitations.

### Description

The galimplant® interfaces for multi-position straight aesthetic abutments are made from grade 5 titanium and can be used in galimplant® multi-position straight aesthetic abutment with different connection types (internal and external), platform and height, multi-position straight aesthetic abutment with metric 2 and angled aesthetic multi-position abutment with metric 2.

The interfaces for galimplant® aesthetic straight multi-position abutments are available with narrow (3.8mm) and wide (4.8mm) platforms, both with rotational (pink anodized) and anti-rotational (gold anodized) configurations and heights 4, 7 and 7.5mm, the latter configuration being cut to 6 and 4.4mm.

Supplied with 2 screws blue colour anodized (one clinical and another prosthetic).

Recommended torque 25 Ncm.

Do not re-use. Single use.

### Precautions during use

- Tissue reaction caused by provisional cement.
- Difficulties removing remaining cement.
- Less control of the crown or bridge's settling during the cementing process.

See the following table for the prosthetic abutment models compatible with each of the configurations.

CONFIGURATIONS					
<i>PITEMU: unit sale // EPITEMU: kit sale</i>					
Rotational		Non-rotational		Transformable to External	
Narrow	Wide	Narrow	Wide		
PITEMUR 4040 PITEMUR 4070 PITEMUR DA4040 PITEMUR DA4075 EPITEMUR 4040 EPITEMUR 4070 EPITEMUR DA4040 EPITEMUR DA4075	PITEMUR 5040 PITEMUR 5070 PITEMUR DA5040 PITEMUR DA5075 EPITEMUR 5040 EPITEMUR 5070 EPITEMUR DA5040 EPITEMUR DA5075	PITEMUA 4040 PITEMUA 4070 PITEMUA DA4040 PITEMUA DA4075 EPITEMUA 4040 EPITEMUA 4070 EPITEMUA DA4040 EPITEMUA DA4075	PITEMUA 5040 PITEMUA 5070 PITEMUA DA5040 PITEMUA DA5075 EPITEMUA 5040 EPITEMUA 5070 EPITEMUA DA5040 EPITEMUA DA5075	PITEMUA 40T PITEMUA 70T PITEMUA DAE4075 PITEMUA DAE4075 EPITEMUA 40T EPITEMUA 70T EPITEMUA DAE4075 EPITEMUR DAE4075	
Compatible Multi-position Abutments Models					
Aesthetic Straight Multi-position / Aesthetic Straight Multi-position, Metric 2 / Angled Aesthetic Multi-position, Metric 2			Aesthetic Straight Multi-position / Aesthetic Straight Multi-position, Metric 2		
#	MUSCE MUSCI MUSCI M2 EMUCI M2		#	MUSCE MUSCI MUSCI M2	

Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.	Non-rotational aesthetic straight multi-position abutment with narrow platform.	Non-rotational aesthetic straight multi-position abutment with wide platform.	Transformable to external aesthetic straight multi-position abutment
Rotational slim aesthetic straight multi-position abutment.	Angled aesthetic multi-position abutment, metric 2	Rotational slim aesthetic straight multi-position abutment.	Non-rotational aesthetic straight multi-position abutment of metric 2 with wide platform.	

## Screw-retained Prosthesis Base

### Intended use

Our screw-retained prosthesis bases with direct implant connection over which the crown is cemented in the laboratory, are designed for definitive screw-retained restorations with the aim of restoring the patient's mastication function and phonetics.

### Indications

The use of screw-retained prosthesis bases is recommended in fixed single- or multi-unit screw-retained restorations.

### Description

The galimplant® screw-retained prosthesis base is made from grade 5 titanium and can be used in galimplant® implants with different connection types (internal and external), platforms, heights and lengths.

Our galimplant® screw-retained prosthesis bases are available in the following configurations:

- Internal Connection:
  - Rotational screw-retained prosthesis bases and pink colour anodized.
  - Non-rotational screw-retained prosthesis bases and golden colour anodized.
- External Connection:
  - Non-rotational screw-retained prosthesis bases and golden colour anodized.

Supplied with 2 screws blue colour anodized (one clinical and another prosthetic).

Recommended torque 30 Ncm.

Do not re-use. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS		External Connection	Internal Connection		
		PZ: unit sale // EPGZ: kit sale			
Platform	Height	Non-rotational	Rotational	Non-rotational	
Narrow	1mm	PZA 40 EPGZA 40	PZR 04010 EPGZR 04010	PZA 04010 EPGZA 04010	
	2mm	-	PZR 04020 EPGZR 04020	PZA 04020 EPGZA 04020	PZA 040207 EPGZA 040207
	3mm	-	PZR 04030 EPGZR 04030	PZA 04030 EPGZA 04030	PZA 040307 EPGZA 040307
	4mm	-	PZR 04040 EPGZR 04040	PZA 04040 EPGZA 04040	PZA 040407 EPGZA 040407
	5mm	-	PZR 04050 EPGZR 04050	PZA 04050 EPGZA 04050	PZA 040507 EPGZA 040507
Wide	2mm	-	-	PZA 05020 EPGZA 05020	PZA 050207 EPGZA 050207
	3mm	-	-	PZA 05030 EPGZA 05030	PZA 050307 EPGZA 050307
	4mm	-	-	PZA 05040 EPGZA 05040	PZA 050407 EPGZA 050407
	5mm	-	-	PZA 05050 EPGZA 05050	PZA 050507 EPGZA 050507
Compatible Implant Model		ICE IPXE		ICI IPX	

CONFIGURATIONS			
PCERCMU: unit sale // EPCERCMU: kit sale			
Rotational		Non-rotational	
Narrow	Wide	Narrow	Wide
PCERCMMUR 40 EPCERCMMUR 40	PCERCMMUR 50 EPCERCMMUR 50	PCERCMMUA 40 EPCERCMMUA 40	PCERCMMUA 50 EPCERCMMUA 50
Compatible Multi-position Abutments Models			
Aesthetic Straight Multi-position Abutment			
#	MUSCE MUSCI		
Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.	Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.
Rotational slim aesthetic straight multi-position abutment.		Rotational slim aesthetic straight multi-position abutment.	

## Cast-to Abutments

### Intended use

Our cast-to abutments with a machined base are designed for definitive screw-retained restorations with the aim of restoring the patient's mastication function and phonetics.

### Indications

Our cast-to abutments are recommended in single or multi-unit restorations when a cast of the most coronal part is necessary.

### Description

Galimplant® cast-to abutments can be used in galimplant® implants with different connection types (internal and external), platforms, diameters and lengths.

Our cast-to abutments are available in the following configurations: rotational and non-rotational. **They are to be used specifically for the elaboration of dental crowns in the prosthetic laboratory.**

Supplied with 2 screws blue colour anodized (one clinical and another prosthetic). The recommended torque for the prosthetic screw is 30 Ncm.

Do not re-use. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS			
External Connection		Internal Connection	
Rotational	Non-rotational	Rotational	Non-rotational
EPSR 40	EPSA 40	EPSR 040	EPSA 040
Compatible Implant Model			
ICE IPXE		ICI IPX	

## Cast-to Abutment for Multi-position Straight Aesthetic

### Intended use

Our cast-to abutments for galimplant multi-position straight aesthetic abutments with a machined base are designed for definitive screw-retained restorations with the aim of restoring the patient's mastication function and phonetics.

### Indications

Our cast-to abutments for galimplant multi-position straight aesthetic abutments are recommended in single or multi-unit restorations when a cast of the most coronal part is necessary.

### Description

The cast-to abutment for galimplant® multi-position straight aesthetic abutments can be used in galimplant® multi-position straight aesthetic original abutments with different connection, platform and height types

Our cast-to abutments for galimplant multi-position straight aesthetic abutments are available in the following configurations: rotational and non-rotational. **They are to be used specifically for the elaboration of dental crowns in the prosthetic laboratory.**

Supplied with 2 screws blue colour anodized (one clinical and another prosthetic). The recommended torque for the prosthetic screw is 30 Ncm.

Do not re-use. Single use.

CONFIGURATIONS			
PSMU: unit sale // EPSMU: kit sale			
Rotational		Non-rotational	
Narrow	Wide	Narrow	Wide
PSMUR 4048 EPSMUR 4048	PSMUR 5048 EPSMUR 5048	PSMUA 4048 EPSMUA 4048	PSMUA 5048 EPSMUA 5048
Compatible Multi-position Abutments Models			
Aesthetic Straight Multi-position Abutment			
#	MUSCE MUSCI		
Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.	Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.
Rotational slim aesthetic straight multi-position abutment.		Rotational slim aesthetic straight multi-position abutment.	

## Overdent Abutments

### Intended use

The overdent abutments with direct implant connection are intended to retain and support the detachable prosthesis, with the aim of restoring the patient's mastication function and phonetics.

### Indications

The use of overdent abutments is recommended in the following cases:

- Supported-implant prostheses
- Retained-implant prostheses

They enable the correction of the implants' angulation of up to 10 degrees.

### Description

Galimplant® overdent abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (internal and external), platforms, diameters and lengths.

Our overdent abutments are available for internal and external connections with shoulder heights of 0.8, 1.6, 3, 4, 5 and 7 mm.

The shoulder height will depend on the soft tissue height. In the case of divergent implants, we can resolve non-parallelism up to an angulation of 10° with this abutment.

Supplied with:

- 1 cap
- 1 separator ring
- 4 retainers, red for exclusive use in the laboratory and white, light blue and dark blue for clinical use, ordered from highest to lowest retention.

Recommended torque 30 Ncm. Only the galimplant® overdent wrench should be used.

Do not re-use. Single use.

### To choose the height of the overdent abutment

- Measure the height of the tissue from the implant platform up to the highest point of the gingival crest (or tissue) surrounding the implant.
- Choose the corresponding abutment which has exactly the same measurement as the tissue, or the next highest measurement, never lower.
- The overdent abutment has a reference mark around the whole abutment which should be on the same level as the gum.

### Recommendations

- Periodically replace the retainers. To ensure proper maintenance of the prosthesis, the clinician must review the retention of the prosthesis every 6 months and replace the retainers if necessary.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS		
Height	External Connection	Internal Connection
0,8 mm	KO 4008	KO 04008

1,6 mm	KO 4016	KO 04016
3 mm	KO 4030	KO 04030
4 mm	KO 4040	KO 04040
5 mm	KO 4050	KO 04050
7 mm	KO 4070	KO 04070
Cap	TKO 4048	TKO 4048
Compatible Implant Model	ICE IPXE	ICI IPX

## Ball Abutments

### Intended use

Our ball abutments with direct implant connection are intended for retained-implant or supported-implant prostheses with the aim of restoring the patient's mastication function and phonetics.

### Indications

The use of ball abutments is recommended in the following cases:

- Supported-implant prostheses
- Retained-implant prostheses

They enable an angulation correction of up to 30 degrees.

### Description

Galimplant® ball abutments are made from grade 5 titanium and can be used in galimplant® internal connection implants with different platforms, diameters and lengths.

Our ball abutments are available with the following shoulder heights: 0.8, 1.6, 3, 4 and 5 mm.

Their height will depend on the soft tissue height.

In the case of divergent implants, with this abutment we can resolve non-parallelism up to an angulation of 30°.

Supplied with:

- 1 cap
- 1 O-ring

Recommended torque 30 Ncm.

Do not re-use. Single use.

### Recommendations

- Periodically replace the retaining bond.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS	
Height	Internal Connection
0,8 mm	KB 04008
1,6 mm	KB 04016
3 mm	KB 04030
4 mm	KB 04040
5 mm	KB 04050
Cap	TKB 4048
Compatible Implant Model	IPX ICI

### Contraindications

The use of prosthetic abutments is not recommended for patients:

- who do not meet the necessary medical requirements for dental implant surgery.
- who are allergic or hypersensitive to commercially pure titanium or Ti-6Al-4V titanium alloy (titanium, aluminium and vanadium).

### Side effects, interactions and complications

The side effects, interactions and complications are those derived from implant placement procedures.

Possible complications following prosthetic restoration include:

- Complications: Loosening/fracture of the abutment and/or screw, mechanical problems associated with the prosthesis, loss of prosthesis retention, loss of the hole sealing of the screw-retained prosthesis, microgap in the implant-abutment connection and microleakage.

If one of these disorders/complications appears, the patient should consult their specialist doctor who will give them the corresponding instructions.

### Storage and transport

The prosthetic abutments are delivered packaged in two different ways:

- in peel-pack bags inside a closed cardboard blister on which the label is placed and includes all product information, including the date of manufacture, which can be deduced from the batch number, the first four digits corresponding to the year and the next two to the month of manufacture (YYYYMMXXXX).
- inside a sealed plastic blister pack, on whose lid all the product information is printed, including the manufacturing date, which can be deduced from the batch number, with the first four digits corresponding to the year and the following two to the month of manufacture (YYYYMMXXXX).

This packaging contains and protects the product during storage and transport and does not have a specified shelf-life.

Prosthetic abutments must be stored in a dry place in the original packaging at room temperature and not exposed to direct sunlight.

When removing the product from its packaging one must verify that it is not damaged and that the product fits the description on the label.

**Caution!** All our prosthetic abutments are sold in a NON-STERILE state.

Before use, they must be cleaned, disinfected and sterilised, observing the procedures validated by Nueva Galimplant SLU, included in the Cleaning/Disinfection/Sterilization section. Said procedures should be carried out before intraoral use. The repetition of the procedures described in this section do not alter the characteristics of these devices. The non-observance of these instructions may cause cross infections.

### Cleaning/Sterilisation/Sterilization

The use of neutral or slightly alkaline cleaning agents is recommended. It is essential to verify that the cleaning product used is compatible with the medical devices are going to processed and respect the specifications of use (temperature, concentration and time) indicated by the manufacturer.

### Cleaning and disinfection

To clean and disinfect the attachments either of the two following methods may be used:

#### Manual Method

- Place the attachments in a perforated tray or basket.
- Prepare an ultrasonic bath and a detergent and disinfectant solution in accordance with the manufacturer's specifications of the product used. *Nueva Galimplant S.L.U. has validated this process using the product neodisher® septo PreClean diluted at 1% to prepare the ultrasonic bath.*
- Immerse the basket with the attachments in the ultrasonic bath and keep them at a temperature of 20° C.
- Remove the attachments from the ultrasonic bath and rinse with plenty of purified water.
- If necessary, repeat the cleaning and disinfection step.

#### Automatic Method

- Place the attachments in the thermal washer-disinfector's tray or basket. Nueva Galimplant S.L.U. recommends thermal washer-disinfectors that comply with the EN ISO 15883 normative.
- Run the cleaning, disinfecting and drying cycle. *Nueva Galimplant S.L.U. has validated the Vario-TD program by the professional brand Miele, using the detergent neodisher® MediClean forte diluted at 0.5%. This program consists of:*
  - Cold water soak for 4 minutes.
  - 55° C clean for 10 minutes with detergent.
  - Cold water neutralisation for 6 minutes.
  - Cold water mid-cycle rinse for 3 minutes.
  - 93° C thermal disinfection for 5 minutes.
  - 110° C dry for 25 minutes.

To guarantee optimum protection of the attachments we recommend that purified water is used throughout the whole process.

Note: it is important to always follow the instructions provided by the detergent and thermal washer-disinfector manufacturers.

### Packaging

Once they are clean, disinfected and completely dry, the attachments must be packaged in a sterilisation bag.

The packaging of thermally sterilised medical devices must comply with the following requisites:

- Be appropriate for steam sterilisation, that is, have a minimum thermic resistance of 137° C and be sufficiently permeable for steam.
- Protect the devices sufficiently from mechanical damage.
- Comply with the EN ISO 11607 normative.

### Sterilisation

- Load the autoclave with the conditioned attachments in the sterilisation bags. Do not overload the autoclave and always follow the equipment manufacturer's instructions.
- Just like in the other steps of these instructions, we recommend the use of purified water to guarantee optimum quality.
- Nueva Galimplant S.L.U. recommends using an autoclave which complies with the requisites of the EN ISO 17665 normative, in addition to being validated and having adequate maintenance.

## Screw-retained Prosthesis Base for Aesthetic Straight Multi-position Abutments

### Intended use

Our screw-retained prosthesis bases for galimplant® aesthetic straight multi-position abutments with direct implant connection over which the crown is cemented in the laboratory, are designed for definitive screw-retained restorations with the aim of restoring the patient's mastication function and phonetics.

### Indications

The use of screw-retained prosthesis bases for galimplant® aesthetic straight multi-position abutments is recommended in fixed single- or multi-unit screw-retained restorations.

### Description

The galimplant® screw-retained prosthesis base for galimplant® aesthetic straight multi-position abutments are made from grade 5 titanium and can be used in galimplant® implants with different connection types (internal and external), platforms, diameters and lengths.

Our galimplant® screw-retained prosthesis bases for galimplant® aesthetic straight multi-position abutments are available for rotational (pink colour anodized) or non-rotational (golden colour anodized) configurations.

Supplied with 2 screws blue colour anodized (one clinical and another prosthetic).

Recommended torque 30 Ncm.

Do not re-use. Single use.

See the following table for the prosthetic abutment models compatible with each of the configurations.

CONFIGURATIONS				
PZMU: unit sale // EPGZMU: kit sale				
Rotational		Non-rotational		Transformable to External
Narrow	Wide	Narrow	Wide	
PZMUR 4040 PZMUR 4070 EPGZMUR 4040 EPGZMUR 4070	PZMUA 5040 PZMUR 5070 EPGZMUR 5040 EPGZMUR 5070	PZMUA 4040 PZMUA 4070 EPGZMUA 4040 EPGZMUA 4070	PZMUA 5040 PZMUA 5070 EPGZMUA 5040 EPGZMUA 5070	PZA 40T PZA 70T EPGZA 40T EPGZA 70T
Compatible Multi-position Abutments Models				
Aesthetic Straight Multi-position Abutment				
#	MUSCE MUSCI			
Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.	Rotational aesthetic straight multi-position abutment with narrow platform.	Rotational aesthetic straight multi-position abutment with wide platform.	Transformable to external rotational aesthetic straight multi-position abutment
Rotational slim aesthetic straight multi-position abutment.		Rotational slim aesthetic straight multi-position abutment.		

## Compatible Interfaces with Cerec System

### Intended use

The compatible interfaces with Cerec System are intended for use with Cerec system with the aim of restoring the patient's mastication function and phonetics.

### Indications

The compatible interfaces with Cerec System allow the realization of unitary cemented-screwed restorations achieving an optimal fit with the implant.

### Description

The compatible interfaces with Cerec System are made from grade 5 titanium and can be used in galimplant® internal connection implants, with different platforms, diameters and lengths.

The compatible interfaces with Cerec System are available in golden colour anodized and Internal Connection for heights of 1, 2, 3 and 4 mm.

Supplied with 2 screws blue colour anodized (one clinical and another prosthetic).

Recommended torque 30 Ncm.

Do not re-use. Single use.

See the following table for the implant models compatible with each of the configurations.

CONFIGURATIONS	
PCERC: unit sale // EPCERC: kit sale	
Height	Internal Connection
1mm	PCERC 040 EPCERC 040
2mm	PCERC 04020 EPCERC 04020
3mm	PCERC 04030 EPCERC 04030
4mm	PCERC 04040 EPCERC 04040
Compatible Implant Model	ICI IPX

## Compatible Interfaces with Cerec System for Aesthetic Straight Multi-position Abutments

### Intended use

The compatible interfaces with Cerec System for galimplant® aesthetic straight multi-position abutments are intended for use with Cerec system with the aim of restoring the patient's mastication function and phonetics.

### Indications

The compatible interfaces with Cerec System for galimplant® aesthetic straight multi-position abutments allow the realization of unitary cemented-screwed restorations achieving an optimal fit with the implant.

### Description

The compatible interfaces with Cerec System for galimplant® aesthetic straight multi-position abutments are made from grade 5 titanium and can be used in galimplant® internal connection implants, with different platforms, diameters and lengths.

The compatible interfaces with Cerec System for galimplant® aesthetic straight multi-position abutments are available in rotational (pink anodized) or anti-rotational (gold anodized) configurations.

Supplied with 2 screws blue colour anodized (one clinical and another prosthetic).

Recommended torque 30 Ncm.

Do not re-use. Single use.

See the following table for the prosthetic abutment models compatible with each of the configurations.

The validation of the sterilisation was carried out in a steam autoclave using a 134° C pre-vacuum cycle for 6 minutes.

- To control the sterilisation process, the use of chemical indicators is recommended, such as, for example, the verification strips which the sterilisation bags come equipped with and which change colour when the sterilisation process has finished.

Note: it is important to always follow the autoclave manufacturer's instructions and keep maintenance up to date.

**Caution!** The re-use and/or re-sterilisation of prosthetic abutments can cause risk of infection and/or cross contamination.

**Documentation and traceability.**

Nueva Galimplant S.L.U. recommends keeping an exhaustive clinical, radiological and photographic documentation. All products can be identified through their reference and batch code and / or UDI code.

**WARNINGS**

Only original prosthetic abutments should be placed in the galimplant® implants. The prosthetic and clinical practitioner must verify that the materials and techniques to be used during prosthetic and clinical procedures are appropriate and compatible with the galimplant® prosthetic abutments and must follow the manufacturer's instructions for use of these materials. Before the prosthetic part, the physician should carry out an exhaustive patient study, in order to determine a correct planning of the case. For the placement of the prosthetic abutments, use only original wrenches of the brand galimplant®. Consult the compatible product models in the catalogue available at [www.galimplant.com](http://www.galimplant.com). There is a risk of aspiration or ingestion of the products when they are used, for this reason, they should only be used by professionals. The prosthetic abutments must solely and exclusively be placed during the implants osteointegration or if the implants have a great primary stability. Failure to observe the procedures set out in these instructions could harm the patient and give rise to one or all of the following complications:

- Aspiration or swallowing of the components.
- Infection.
- Damage to the implant, abutment, components or instruments.
- Implant failure.

To guarantee a good long term treatment result, the clinician must inform the patient about appropriate home oral hygiene and the importance of attending periodic clinical and radiographic consultations to perform a control and preventive diagnosis of any side effect and/or complication.

In the event of a serious incident related to the use or result of use of the product, inform Nueva Galimplant ([www.galimplant.com](http://www.galimplant.com)) and your National Competent Authority.

The practitioner must provided the patient with the information about side effects, interactions and complications in relation to galimplant® prosthetic abutments.

**Procedures for the disposal of the product**

Once they have been extracted from the oral cavity, the prosthetic components must be disposed of as biological waste, in accordance with current local laws.

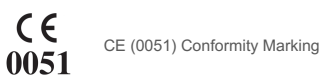
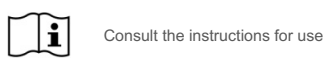
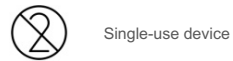
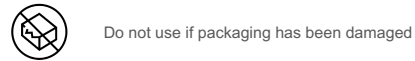
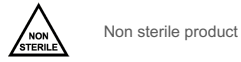
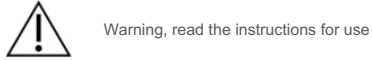
**Summary of Safety and Clinical Performance**

The Summary of Safety and Clinical Performance (SSCP) for the product is available in the European database EUDAMED. The document can be obtained from the website: <https://ec.europa.eu/tools/eudamed>.

**Validity**

These instructions for use will be valid as of February 2026.

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